

MISTY FAST® & MISTY MAX 10® SMALL VOLUME NEBULIZERS

Cleaning Instructions

Suggested Cleaning Instructions for Hospital Use

AirLife® Misty Fast® and Misty Max 10® small volume nebulizers are single-patient, multiple-use, and disposable. The nebulizers should be replaced after 24 hours of cumulative use or 50 cleaning cycles, whichever is first. Replace the nebulizer sooner if it becomes damaged, contaminated, or is not functioning properly.

Cleaning procedures may vary depending on the patient population, disease process, and the hospital's infection control policies. To clean the Misty Fast or Misty Max nebulizers, follow the instructions or use your facility's approved cleaning procedure.

Cleaning Method

1. After each use, unscrew the nebulizer cap and bottle.
2. Remove the one-piece jet by pulling and twisting the jet off the jet stem.
3. Wash all components in warm, soapy water and rinse well. Sterile water may also be used in place of soapy water.
4. Shake out/off excess water or sterile water and air dry for 30 minutes. Alternatively, shake out/off excess water or sterile water and wipe the device with a clean, lint-free cloth until dry. To aid in the drying process, you may also use the compressed air supply to clear the orifice.
5. Reseat the jet by placing the jet over the jet stem and snap into place.
6. Reattach the nebulizer cap and bottle.
7. Cumulative duration of use not to exceed 24 hours or 50 cleaning cycles.

Always follow your facility's policy and procedure for the delivery of aerosol treatments and the cleaning of equipment.