## Gas Sampling Lines

**EN**
**DEVICE DESCRIPTION:** The device is non-sterile, disposable, single-patient use sampling line that enables sampling of exhaled gases. The sampling line available in different end connectors.

**INTENDED PURPOSE:** The Sample Lines are intended to be used where exhaled gas is monitored.

**INDICATIONS FOR USE:** The Sample Lines are intended to be used to sample exhaled gas in intubated or non-intubated patients requiring expired gas monitoring.

**ENVIRONMENT:** Hospital, sub-acute, pre-hospital.

**PATIENT TARGET GROUP:** Infant, pediatric, adult.

**EXPECTED CLINICAL BENEFITS:** The CO2 gas sampling line: Capnography provides early detection and warning of changes in a patient’s ventilatory status. This enables to clinicians to adjust medication dosage or apply therapeutic intervention to prevent hypoventilation. ETCO2 monitoring provides informaiton on endotracheal tube placement, the effectiveness of chest compressions and the return of circulation.

**CONTRAINDICATIONS:** None Known.

**WARNINGS:**

- If oxygen is in use, do not use near flame or heat source.
- Position tubing to avoid strangulation.
- Patient may become hypoxic if oxygen flow is interrupted.
- Keep excess tubing loosely coiled and out of the way to prevent kinking and tripping hazard.
- To reduce the risk of misconnections and patient injury, always trace tubing from gas source to the medical device before connecting.

**CAUTIONS:**

- Consult physician if patient develops infection, skin irritation or material sensitization.
- Do not place anything on supply tubing that may obstruct flow.
- Excessive patient secretion or a build-up of liquids in the tubing may occlude the sampling and require more frequent replacement.

**RESIDUAL RISKS:** Refer to Warnings and Cautions.

**NECESSARY USER QUALIFICATIONS:** This device is restricted to sale by or on the order of a physician. There are no additional requirements for special facilities, special training, or particular qualifications for the use of this device.

**INSTRUCTIONS:** Follow instructions to avoid risk of infection and contamination.

- Connect the gas sampling line to the capnography monitor.
- Connect the other end of the gas sampling line to the luer connector of the circuit adaptor or sampling line.
- Check the monitor display for EtCO2 value and/or waveform.

This product is disposable and is not intended for prolonged use greater than 7 days.

**SAFE DISPOSAL:** Dispose of device in accordance with local, state or national regulations.

**INCIDENT REPORTING:** Contact your country’s Competent Authority and Manufacturer to report any serious incident.

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