

# Avanos\* Neonatal and Pediatric Closed Suction System



## Pediatric Endotracheal

Measured Depth Suction Card

My name is: \_\_\_\_\_

Intubation Date: \_\_\_\_\_

ET Tube Size: \_\_\_\_\_

Suction Catheter Size: \_\_\_\_\_

CIRCLE THE SUCTION DEPTH  
ON THE CATHETER PICTURE  
FOR REFERENCE.

Color bands allow easier  
visualization on subsequent  
suction procedures.



II  
(Observation area)

## Endotracheal



### To Determine Suction Depth

(observe in area directly across from lavage port connector)

Number on ET tube nearest endotracheal tube adapter tip \_\_\_\_\_ cm

Add 5.0 cm to suction **at ET tube tip** + \_\_\_\_\_ cm

Add 5.5 cm to suction .5 cm **at ET tube tip**

Equals suction depth: Total \_\_\_\_\_ cm