



BALLARD* 24-HOUR ORAL CARE KIT | Q2

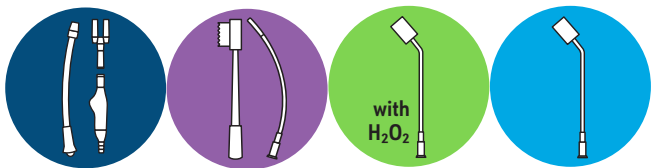


1

BALLARD* Prep Pack




2

BALLARD* Toothbrush and Suction Catheter (14 Fr) Packs




4

BALLARD* Suction Swab Packs with H₂O₂

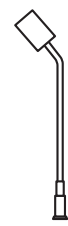

6


BALLARD* Suction Swab Packs with Alcohol-Free Mouthwash

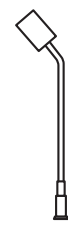

Instructions for Use: Oral care should be performed per institutional protocol.** For single patient use. Patients with altered levels of consciousness or who cannot comprehend commands may require use of a bite block. See instructions for use below.

PREP PACK ASSEMBLY

- Select BALLARD* Prep Pack from kit and open package.
- Connect stepped adapter of suction handle firmly to suction canister tubing. Attach Y-adapter firmly to suction port. Then firmly attach oral care suction tubing to one Y-adapter port and closed suction tubing to the other port. (Use the Y-adapter only if using a single suction canister for both closed suctioning and oral care.)
- Attach Yankauer firmly to suction handle.
- To store between uses, clip suction handle to top of suction tubing in upright position.

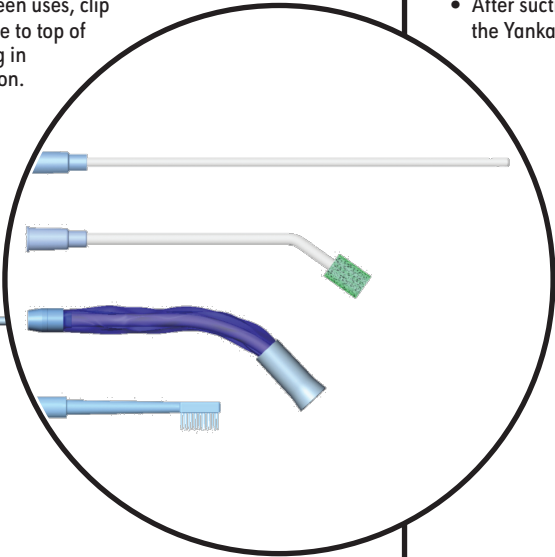

YANKAUER ASSEMBLY AND USE

- Attach Yankauer firmly to suction handle.
- Slide cleaning sleeve toward suction handle.
- To apply suction, slide switch to ON.
- Suction and clean oral cavity.
- After suctioning, advance cleaning sleeve toward tip to clean and dry the Yankauer.
- With suction still on, immerse Yankauer tip in sterile water or appropriate solution to clear Yankauer and suction tubing.
- Slide switch to OFF. (Switch should remain in OFF position when Yankauer is not in use.)
- To store, clip suction handle to top of suction tubing in upright position.
- Note:** discard tubing, Y-adapter, suction handle, and Yankauer after using for 24 hours or according to protocol.



TOOTHBRUSH AND SUCTION CATHETER PACK ASSEMBLY AND USE

- Select BALLARD* Toothbrush from kit and open package.
- Remove Yankauer from handle, set aside and attach Toothbrush.
- Peel back solution dispenser lid and moisten Toothbrush.
- Clean and suction oral cavity according to facility protocol.
- To apply suction, slide switch to ON.
- After suctioning is complete, slide switch to OFF.
- Remove Toothbrush from handle and discard. If needed, attach Suction Catheter to handle.
- To apply suction, slide switch to ON.
- Suction and clean oropharyngeal cavity.
- After suctioning is complete, slide switch to OFF.
- Remove Catheter from handle and discard.
- Reattach Yankauer to handle.

- Applicator Swab and Mouth Moisturizer in each pack.**
- Remove applicator swab and coat with moisturizer.
 - Apply moisturizer to lips and mouth.
 - Discard after use.

† Four Suction Swab Packs with H₂O₂ provided for 24-hour period per Drug Facts.

SUCTION SWAB PACK ASSEMBLY AND USE

- Select Suction Swab from kit and open package.
- Remove Yankauer from handle, set aside and attach Suction Swab.
- Peel back solution dispenser lid and moisten swab.
- Clean and suction oral cavity according to facility protocol.
- To apply suction, slide switch to ON.
- After suctioning is complete, Slide switch to OFF.
- Remove Suction Swab from handle and discard.
- Reattach Yankauer to handle.

- Applicator Swab and Mouth Moisturizer in each pack.**
- Remove applicator swab and coat with moisturizer.
 - Apply moisturizer to lips and mouth.
 - Discard after use.

**See graphics on the inner panel of the kit for a recommended order of use of components to support a typical q2h oral care protocol. The Suction Catheter Packs may be used as needed. For convenience, the start time may be entered in the blank arrow.

 Nonsterile	Disposable	 Not made with natural rubber latex	For use by a healthcare professional	Store at room temperature	 97020
--	------------	--	--------------------------------------	---------------------------	--

Mouth Moisturizer Ingredients

Deionized Water, Glycerin, Soybean Oil, Carboxymethylcellulose (CMC), Polysorbate 80, Sorbitol, Vitamin E / Tocopherol Acetate, Spearmint Flavor, Sodium Benzoate, Xylitol, Citric Acid, Sodium Chloride

Alcohol-Free Mouthwash Ingredients

Purified Water, Sorbitol, Glycerin, Flavor, Sodium Saccharin, FD&C Blue #1, FD&C Yellow #5, Cetylpyridinium Chloride, Benzoic Acid, Sodium Benzoate

Oral Debriding Agent	
Drug Facts	
Active Ingredient	Purpose
Hydrogen Peroxide 1.5%.....	Oral Debriding Agent
Uses	
Aids in the removal of phlegm, mucus, or other secretions in the temporary relief of discomfort due to occasional sore throat and sore mouth.	
Warnings	
Stop use and ask a doctor if:	
Swelling, rash, or fever develop.	
Severe or persistent sore throat or sore throat accompanied by high fever, headache, nausea, and vomiting occurs.	
Do not use more than 2 days or administer to children under 3 years of age unless directed by a physician.	
Keep out of reach of children under 3 years of age.	
Directions	
Topical dosage for adults and children 3 years of age and older is a rinse used no more than 4 times daily. For children under 3 years of age, there is no recommended dosage except under the advice and supervision of a dentist or doctor.	
Use only under health care practitioners supervision.	
Other Information	
Store at room temperature.	
Inactive Ingredients	
Purified Water, Glycerin, Flavor, Sodium Saccharin	
Questions or Comments?	1-844-428-2667

Antiplaque Solution	
Drug Facts	
Active Ingredient	Purpose
Cetylpyridinium Chloride 0.05%	Antigingivitis/Antiplaque
Uses	
Helps remove plaque that leads to gingivitis.	
Warnings	
Stop use and ask a doctor if:	
Gingivitis, bleeding, or redness persists for more than 2 weeks.	
You have painful or swollen gums, pus from the gum line, loose teeth, or increasing spacing between the teeth. These may be signs or symptoms of periodontitis, a serious form of gum disease.	
Keep out of reach of children under 6 years of age.	
If more than used for rinsing is accidentally swallowed, get medical help or contact a Poison Control Center right away.	
Directions	
Adults and children 12 years of age and older: Use as directed by a dentist. Do not swallow the rinse.	
Children 6 years to under 12 years of age: supervised use.	
Children under 6 years of age: do not use.	
Use only under health care practitioner's supervision.	
Other Information	
This rinse is not intended to replace brushing or flossing.	
Store at room temperature.	
Inactive Ingredients	
Purified Water, Glycerin, Sorbitol, Flavor, Benzoic Acid, Sodium Benzoate, Sodium Saccharin, D&C Green 5	
Questions or Comments?	1-844-428-2667