AirLife®

Vital Signs® Face Mask

Size	Patient Population
0	Preemie
1	Neonate
2	Infant
3	Toddler
4	Child/Small Adult
5	Adult
6	XL Adult

EN

This product is not sterile. It is made of standard medical device materials and is a single patient disposable product. This product should not be used if physical deterioration, brittleness or discoloration occurs. Good clinical practice requires proper stock rotation to prevent older product from being stored for an excessive period of time.

Face mask with adjustable air cushion

Directions for use

For masks with a tail inflation valve:

1. Insert a luer tipped syringe into the valve, squeeze valve "O" ring and inflate/deflate.
2. In an emergency Squeeze Valve "O" Ring and inflate by mouth.

For masks with a top valve:

1. Insert a luer tipped syringe into the valve and inflate/deflate mask cushion.

Placement of Mask:

1. Place mask cushion on the patient's chin, roll cushion up the face with a gentle but firm pressure to create an effective mask seal.
2. Refrain from pressing rigid part of the mask against the face, nose or eyes.
3. A head strap can be affixed to the optional prong ring or detached when not needed.

WARNING

Do not use in the presence of flammable anaesthetics. Do not rinse, soak, wash or sterilize.

CAUTION

Excessive pressure applied to face mask can lead to facial and/or optic nerve damage.

Single Use Devices:

This single-use product is not designed or validated to be reused. Reuse may cause a risk of cross-contamination, affect the measurement accuracy, system performance, or cause a malfunction as a result of the product being physically damaged due to cleaning, disinfection, re-sterilization, or reuse.

Some of the devices covered by these Instructions for Use may contain phthalates, please refer to additional packaging for phthalate contents.

Risks and precautionary measures related to phthalates:

This instruction pertains to the phthalate symbol marked on the device or its packaging. If this device is used for the treatment of children, or treatment of pregnant or nursing women; please note that the following types of procedures may increase the risk of exposure to phthalates: Exchange transfusion in neonates, total parenteral nutrition in neonates, multiple procedures in sick neonates, hemodialysis in peripubertal males, male foetus and male infant of pregnant women, and lactating women; and massive blood transfusion into trauma patients. Although these procedures have the potential for increased risk of exposure, conclusive evidence of human health risks has not been established. As a precautionary measure, to reduce the potential for unnecessary exposures to phthalates, the product must be used in accordance with the instructions for use, and practitioners should refrain from using this product beyond the period of time the product is medically necessary or needed.

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AirLife Australia Holdings Pty Ltd