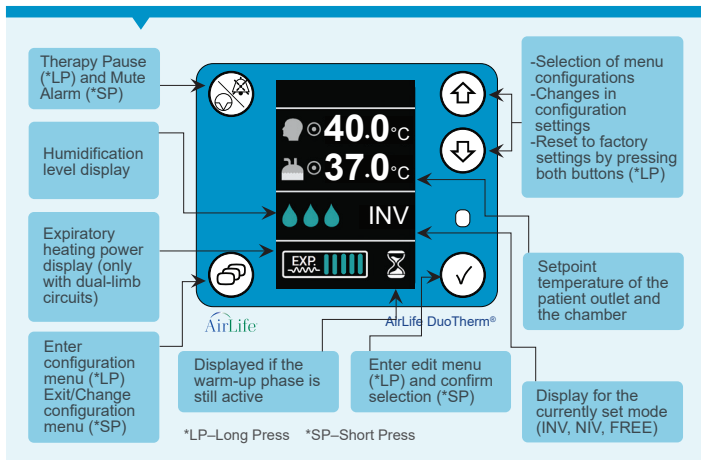




AirLife DuoTherm® Humidification System Quick Guide



Configuration help




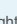


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
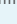
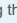

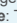

Attention! This quick guide does NOT replace the operating instructions. Specifications provided by the doctor or hospital take priority over these recommendations.

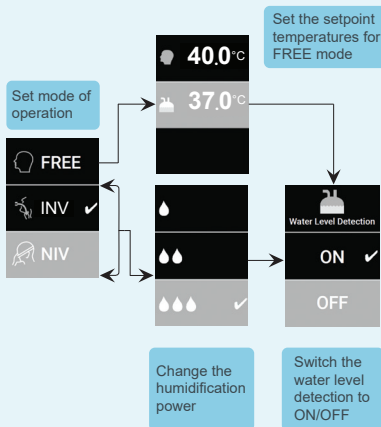
All warnings and guidelines in the operating instructions must be strictly observed.

- “INV” mode: Factory setting for invasively ventilated patients (setpoint temperature near the patient = 40°C; chamber = 37°C max.)
- “NIV” mode: Factory setting for mask-ventilated patients (setpoint temperature near the patient = 34°C; chamber = 31°C max.)
- “FREE” mode: Enter individual operating parameter within +2 to -5 restrictions
- Light condensate in the patient tube = Sufficient humidification
- No condensate = Increase humidification
- Too much condensate = Reduce humidification
- Humidification can be adjusted by water droplets from 1 drop (min.) to 3 drops (max.) or adjusted using the chamber temperature in the FREE mode


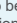



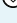


Settings in Configuration menu

Display in the Configuration menu

1. Enter the Configuration menu by pressing  for 3 seconds (*LP).
2. In the Configuration menu, press  to scroll through the parameter selections.
3. Use the  or  buttons to highlight and select parameter settings.
4. Confirm parameter selections by pressing .
5. When the Free mode is selected, follow these steps:
 - a. Use the  or  buttons to highlight the temperature parameter selection.
 - b. Confirm the selection by pressing .
 - c. Change parameter settings by using the  or  buttons.
 - d. Confirm changes by pressing .
 - e. Repeat steps a through d to change other temperature settings (**Note:** The system has a +2 to -5 temperature settings restrictions).
6. Exit Configuration menu by pressing  SP.



Settings in edit menu

1. Enter the Edit menu by pressing the  button for 3 seconds (*LP).
2. Select the setting to be changed by pressing either the  or  buttons.
3. Confirm your selection by pressing the .
4. Increase or decrease the selection by using the  or  buttons.
5. Confirm entered value by pressing the .
6. Repeat steps 2 through 5 to change other selections.
7. Exit the Edit menu by pressing  for 3 seconds (*LP).
8. Expiratory heating power can be adjusted in Edit mode.

Display in the edit mode

