

REF 0512

UNITS - 1

RX Only



GAS SAMPLING LINE

SONDA DE MUESTREO DE GASES

LINHA DE RECOLHA DE AMOSTRAS DE GÁS

LINEA DI CAMPIONAMENTO DEL GAS

TUBULURE D'ÉCHANTILLONNAGE DE GAZ

GAS-PROBENNAHME-SCHLAUCHLEITUNG

10' PVC, .050" ID x .108" OD, M/M Luers



CAUTION: Use of sterilization or cleaning solutions may leave this device nonfunctional.

SUGGESTED USE:

1. Attach sample line male luer lock connector to female luer connector on the patient end of the circuit. If swivel luer is present, adjust the elbow to desired position prior to securing.
2. Attach other end (male or female) to the CO2 monitor.
3. Check for secure connections and proper function prior to use.
4. These devices are not intended for reprocessing. Risks of reprocessing may include microbial contamination or product degradation.

Westmed

5580 S. Nogales Highway
Tucson, Arizona 85706 USA

Phone: 800-975-7987

Fax: 520-294-6061

www.westmedinc.com



MT Promedt Consulting GmbH

Altenhofstr. 80

66386 St. Ingbert, Germany

Assembled in MEXICO

LABEL PN 71270, Rev. 09



LOT