

CO₂/O₂ Mask

Adult Elongated CO₂/O₂ Mask, 10'x 0.50 GLS w/
Swivel Elbow, 2" Pigtail w/Female Luer Connectors,
7' Oxygen Supply Tubing

REF 0562

UNITS - 1

Rx Only



C **E**0482



Not Made
With DEHP



Not Made
With Natural
Rubber Latex



CAUTIONS:

Do not attempt to clean, sterilize or disinfect any part of
the product as this may cause damage to the monitor.

SUGGESTED PROCEDURE:

1. CO₂ Button Luer positioned on the mask for gas sampling.
2. Attach the oxygen tubing to the mask if you desire oxygen delivery, and then connect supply tube to gas source at the desired flow rate.
3. Attach gas sampling line to CO₂ Button Luer on the mask. Adjust the swivel luer elbow to the desired position prior to securing (if applicable).
4. Connect other end of gas sampling line to the CO₂ monitor (if applicable).
5. Discard after single patient use.

Westmed

5580 S. Nogales Highway
Tucson, Arizona 85706 USA
Phone: 800-975-7987
Fax: 520-294-6061
www.westmedinc.com

LOT



MT Promedt Consulting GmbH
Altenhofstr. 80
66386 St. Ingbert, Germany

ASSEMBLED IN MEXICO
LABEL PN 71592, Rev. 05