

REF 0485

Concentrator Humidifier Adapter

15" O₂ Tubing w/ Threaded Nut Connector

UNITS - 1

RX Only



Not Made
With Natural
Rubber Latex



Not Made
With DEHP



CE 0485



CAUTION:

Use of sterilization or cleaning solutions
may leave this device nonfunctional.

SUGGESTED PROCEDURE

1. These products are intended for use in delivering oxygen therapy.
2. All products are to be used in accordance with the applicable prescription.
3. Remove the product from the package
4. The male end of the humidifier connector tube should be screwed into the oxygen outlet port of the humidifier bottle.
Attach other end of tubing to outlet port onto oxygen concentrator.
5. Ensure that all connections are secure, and then start the gas flow.
6. Confirm appropriate function.
7. Discard after single patient use.

Westmed

5580 S. Nogales Highway
Tucson, Arizona 85706 USA
Phone: 800-975-7987
Fax: 520-294-6061
www.westmedinc.com

EC REP

MT Promedt Consulting GmbH
Altenhofstr. 80
66386 St. Ingbert, Germany
MADE IN U.S.A.
LABEL PN 75188, Rev. 04



LOT