5580 S. Nogales Highway • Tucson, AZ 85706 USA • www.myAirLife.com
(If detached, securely attach needle to syringe.) • (All syringes contain ~ 25U of balanced heparin)

English

INSTRUCTIONS FOR USE

CORD GAS KIT

This product is intended for use at the direction of a physician as a single use disposable syringe to obtain arterial or venous blood samples for in vitro diagnostic use only. For pH blood gas and selected whole blood electrolytes. Not for injection. Contents are sterile in unopened, undamaged package.

1. Prepare umbilical cord sample section as per hospital protocol.
2. To ensure smooth operation while aspirating blood, cycle the plunger in the barrel once prior to use.
3. Prepare syringe for sample aspiration by depressing the plunger to the bottom of the syringe and confirming needle is securely affixed to the syringe.
4. Remove protective sheath from needle by pulling straight off, using caution not to damage needle tip.
5. Remove protective sheath from needle by pulling straight off, using caution not to damage needle tip.
6. Hold the syringe at a 45° angle and perform puncture.
7. Aspirate two blood samples (one venous, one arterial) from the excised clamped cord segment.
8. After each sample is aspirated, engage needle protection device by:
 - a. Firm Surface - Use flat firm surface, gently push down on safety device until it engages.
 - b. Thumb - Push safety device with thumb using a forward motion until it engages.
 - c. Fore-finger - Push safety device with forefinger using a forward motion until it engages.
9. Once safety device is engaged, twist device off by grasping needle hub with thumb and forefinger and discard in biohazard container.
10. Attach tip cap to syringe and mix sample thoroughly by rolling syringe between palms for 20-30 seconds.
11. Transport to lab for analysis.

Contains biological material of animal origin.

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