

Muc-Away™ Nasal Aspiration Device

Indications for Use:

Neonatal nasal aspiration device for removing fluids from the nares.

To Set up


1. Connect barbed end of Muc-Away to suction tubing.
2. Connect other end of suction tubing to suction outlet.
3. Suction pressure should be set per hospital protocol or physician recommendation.

To Aspirate

4. Turn on suction device and occlude Thumb Controller.
5. To stop aspiration, unblock Thumb Controller. (Intermittent suction is accomplished by blocking and unblocking Thumb Controller)
6. Suction sterile water through Muc-Away after each use to clear debris.

Cautions & Warning

- For nasal aspiration use only.
- Caution should be taken when inserting Muc-Away into the nares to avoid tissue trauma.
- Single patient use. Muc-Away should be discarded following hospital protocol.
- US Federal law restricts this device to sale in the USA by or on the order of a physician.

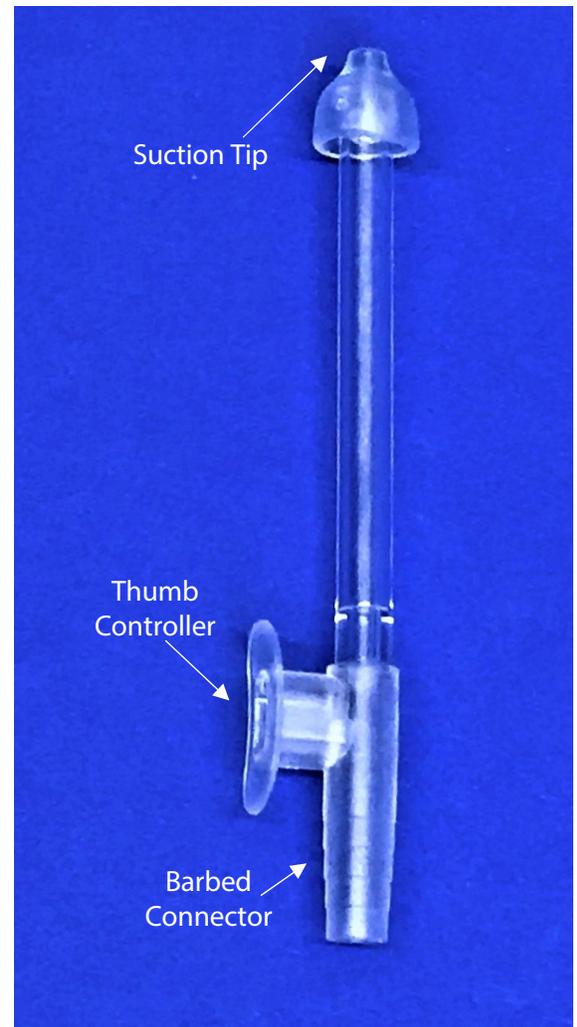

Re-Order No.	Description	QTY/Cs
0488	Muc-Away™ Comfort Soft Plus® Nasal Aspiration Device, "Mini Muc™" 3.25" length	40
0489	Muc-Away™ Comfort Soft Plus® Nasal Aspiration Device, "Mini Muc™" 4.5" length	40
0490	Muc-Away™ Comfort Soft Plus® Nasal Aspiration Device, 3.25" length	40
0491	Muc-Away™ Comfort Soft Plus® Nasal Aspiration Device, 4.5" length	40
0492	Muc-Away™ Nasal Aspiration Device, 3.25" length	40
0493	Muc-Away™ Nasal Aspiration Device, 4.5" length	40

Manufactured by:


Westmed 

5580 S. Nogales Highway
Tucson, Arizona 85706 USA

Phone: 800-975-7987

Fax: 520-294-6061

www.westmedinc.com

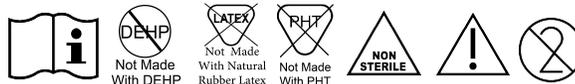

Made in U.S.A
76429, Rev. 05 2021/06