

**REF 0152**

UNITS - 1

LENGTH - 7' (2.1 m)

**Rx Only**



Not Made  
With Natural  
Rubber Latex

COMFORT SOFT PLUS® PREMATURE CANNULA  
CÁNULA PARA PREMATUROS COMFORT SOFT PLUS®  
CÁNULA PARA PREMATUROS COMFORT SOFT PLUS®  
CANULE POUR PRÉMATURÉ COMFORT SOFT PLUS®  
CANNULA PREMaturi COMFORT SOFT PLUS®  
COMFORT SOFT PLUS® KANÜLE FÜR FRÜHGEBORENE

SUGGESTED PROCEDURE:

1. These products are intended for use in delivering oxygen therapy.
2. All products are to be used in accordance with the applicable prescription.
3. Remove the product from the package.
4. Attach the device to the oxygen source and make appropriate connection.
5. Ensure that all connections are secure, and then start the gas flow.
6. Confirm appropriate function prior to use.
7. Discard after single patient use.

**! CAUTION:**

Use of sterilization or cleaning solutions may leave this device nonfunctional.

**Westmed** 


5580 S. Nogales Highway  
Tucson, Arizona 85706 USA

Phone: 800-975-7987

Fax: 520-294-6061

[www.Westmedinc.com](http://www.Westmedinc.com)

Patents Pending

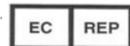

MT Promedt Consulting GmbH  
Altenhofstr. 80  
66386 St. Ingbert, Germany

LABEL PN 77566, Rev. 01

PRODUCT OF MEXICO

ASSEMBLED IN MEXICO

**LOT**