

REF 7-9800

UNITS - 01

Rx Only



CE 0482

DISPOSABLE ANESTHESIA/ RESPIRATORY PRODUCTS

Series 800 Large Adult

Anesthesia Mask

MASCHERA PER ADULTI GRANDE

MÁSCARA GRANDE PARA ADULTO

GROSS MASKE FÜR ERWACHSENE

MASQUE ADULTE - GRAND

SUGGESTED PROCEDURE:

1. Connect to breathing circuit or resuscitation device with appropriate fitting.
2. Pressure check to ensure tight system and inspect all fittings to ensure secure connection.
3. If equipped with an inflation/deflating valve, insert luer tipped syringe into the valve to inflate/deflate cushion.
4. Hand hold or secure with head strap.
5. Discard after use.

⚠ WARNING: Patient should be constantly monitored whenever this device is in use. Do not use in the presence of flammable anesthetics. Do not soak, rinse, wash, gas, or steam sterilize this product. To do so may result in harmful residual gases, or dangerously increase its breathing resistance.



MT Promed Consulting GmbH

Altenhofstr. 80

66386 St. Ingbert, Germany

Westmed

5580 S. Nogales Highway

Tucson, Arizona 85706 USA

Phone: 800-975-7987

Fax: 520-294-6061

www.westmedinc.com

LOT

ASSEMBLED IN MEXICO

LABEL PN 77700, Rev. 01