

**REF 6229**

1 each

**Rx Only**



**C** **€0482**

## **FILTER**

Angled HME w/Sampling Port

22mm OD/15mm ID x 15mm OD

30.63ml Dead space

28.4mg H<sub>2</sub>O/L @ VT 500ml moisture output

0.89cm H<sub>2</sub>O @ 30LPM resistance

0.04cm H<sub>2</sub>O @ 8LPM resistance

### SUGGESTED PROCEDURE:

1. Filters may be fitted at patient end or machine end of circuit.
2. HMEs and HME Filters must be fitted at patient end of circuit.
3. Do not use between the patient and a nebulizer or humidifier.
4. Change the filter as required or as per protocol.

**CAUTION:** For single patient use only. Use of sterilization or cleaning solutions may leave this device nonfunctional. Store in a cool dry place.

**NOTE:** The presence of water droplets and/or salt crystals observed in the HME indicates normal function.

**Westmed**

5580 S. Nogales Highway  
Tucson, Arizona 85706 USA

Phone: 800-975-7987

Fax: 520-294-6061

[www.westmedinc.com](http://www.westmedinc.com)

**EC REP**

MT Promedt Consulting GmbH  
Altenhofstr. 80

66386 St. Ingbert, Germany

Assembled in Mexico

LABEL PN 78097 Rev. 02

**LOT**