

**REF** 7-6205

**UNITS - 1**  
**RX Only**



**Breathing Bag**  
Breathing Bag, 1/2L

**CAUTION:** Use of sterilization or cleaning solutions may leave this device nonfunctional.

**INTENDED USE:** This product is intended for use, at the direction of a physician or other licensed, trained medical personnel as a short term, single use device to be used in the administration of air, oxygen, or other breathing gases to breathing or non-breathing patients.

**TEST CIRCUIT AND ACCESSORIES PRIOR TO USE:**

1. Assemble the circuit and any accessories as required. Attach the assembled system to the breathing machine prior to connecting the patient.
2. Activate gas flow and observe that gas flows freely from the open connecton intended for attachment to the patient. Verify that there are no occlusions. Occlude the open patient connecting port and verify gas flow now moves freely through the exhalatory limb.

**Westmed**

5580 S. Nogales Highway  
Tucson, Arizona 85706 USA  
Phone: 800-975-7987  
Fax: 520-294-6061  
www.westmedinc.com

Assembled in Mexico  
LABEL PN 79325, Rev. 01



MT Promedt Consulting GmbH  
Altenhofstr. 80  
66386 St. Ingbert, Germany  
Patent Pending



**LOT**