Care and Service of
Conventional Laryngoscope Systems

REF

Intended Use: A laryngoscope is a device which helps in intubation during the administration of general anesthesia or for mechanical ventilation.

Care and Service Document: Conventional laryngoscope blades and handles must be processed prior to first use, and between each use, using any of the methods presented in this care and service document:

- CLEANING + (A) HIGH-LEVEL DISINFECTION
- CLEANING + (B) GRAVITY AUTOCLAVE STERILIZATION
- CLEANING + (C) PRE-VACUUM STEAM STERILIZATION
- CLEANING + (D) STERIS AMSCO V-PRO STERILIZATION
- CLEANING + (E) STERRAD STERILIZATION

WARNINGS and CAUTIONS

DO NOT USE THE FOLLOWING: Ultrasonic clean, dry heat sterilization or Potassium Hydroxide liquids.

REMOVE BATTERIES BEFORE cleaning, high-level disinfecting, or sterilizing of the non-waterproof laryngoscope handles.

ENSURE blade is fully engaged on handle prior to use.

CHECK laryngoscope system functionality prior to use.

CLEANING

Laryngoscope Blades & Handles:

Immediately after use, the laryngoscope system should be rinsed under cool running tap water until all visible soil is removed. Rinse hard-to-reach areas with a cleaning gun for at least one (1) minute.

Prior to performing the cleaning procedure, the handle and blade need to be separated and cleaned individually.

Waterproof laryngoscope handles (5-0237-91, 5-0237-92, 5-0237-93, 5-0237-96) can be cleaned without removing the batteries. The waterproof laryngoscope handle can be cleaned using the process below starting with step 1.

Non-waterproof laryngoscope handles (5-0237-01, 5-0237-02, 5-0237-03, 5-0237-05, 5-0237-08) must have the batteries removed prior to cleaning. Disassemble the handle by removing the bottom cap and batteries. The disassembled handle and cap can be cleaned using the process below.

- Immerse the blade, waterproof laryngoscope handle or non-waterproof handle and end cap, in a pre-soak enzymatic cleaning solution, prepared in accordance to the manufacturer's recommendations.
- Brush the hard-to-reach areas of the medical device with a soft brush for at least one (1) minute. Pay attention to hard to reach areas.
- Make sure all the surfaces are completely moistened with the enzymatic cleaning solution.
- Remove the medical device from the enzymatic cleaning solution.
- Rinse the medical device completely under lukewarm running tap water for a minimum of one (1) minute to remove the cleaning solution completely. Check for cleanliness. If dirt is visible, repeat steps 1-5 until visibly clean.
- Dry with a lint free, clean cloth or filtered pressurized air.
Follow with HIGH-LEVEL DISINFECTION or STERILIZATION.

AUTOMATED CLEANING

Automated Cleaning Preparation for Blades: Remove laryngoscope blade from the handle head.

Automated Cleaning Preparation for Handles: Disassemble the laryngoscope handle by removing the end cap and batteries. Set batteries aside.

Load laryngoscope blade and / or separated handle and end cap into an automated washer per ISO 15883 or equivalent and follow the below recommended process chart. Equivalent cycles and detergent can be utilized.

Follow with HIGH-LEVEL DISINFECTION or STERILIZATION.

Phase	Recirculation Time (Minutes)	Temperature	Detergent Type and Concentration
Pre-Wash	02:00	Cold Tap Water	N/A
Wash	02:00	43°C Tap Water	Valsure Enzymatic 2 mL/L
Rinse	01:00	43°C Tap Water	N/A
Dry Time	07:00	90°C	N/A

- 5-0237-XX Conventional Reusable Handles

5-3073-XX Phillips Miller American Profile Blades

5-3078-XX Port Miller American Profile Blades

5-5051-XX Mac English Profile Blades, Clear Bulb

5-5052-XX Mac American Profile Blades, Frosted Bulb

5-5053-XX Mac Blades, Left Handed

5-5057-XX Wis-Hipple Miller American Profile Blades

5-5061-XX Miller English Profile Blades, Clear Halogen
- 5-5062-XX Miller American Profile Blades, Frosted Bulb

5-5077-XX Robert Shaw Miller English Profile Blades

5-5082-XX Sun-Flex™ Mac English Profile Blades

5-5851-XX LED Mac English Profile Blades

5-5852-XX LED Mac American Profile Blades

5-5861-XX LED Miller English Profile Blades

5-5862-XX LED Miller American Profile Blades

(A) HIGH-LEVEL DISINFECTION

Laryngoscope Blades & Handles:

- Prepare the disinfectant solution CIDEX®-OPA (0.55% ortho-Phthalaldehyde based disinfectant) according to the manufacturer's directions.
- Completely immerse the assembled handle or blade in the disinfecting solution. Make sure that all surfaces are completely moistened. Brush the-hard to-reach areas during the exposure time for at least one (1) minute. Allow the device to soak for a minimum of twelve (12) minutes.
- Remove the device from the disinfecting solution.
- Rinse by immersing the device in demineralized water for one (1) minute. Repeat two (2) additional times for a total of three (3) rinse cycles, utilizing fresh demineralized water each time. During the rinsing process, ensure the device is agitated. The handle may need to be disassembled to ensure thorough rinsing.
- Dry with a lint-free clean cloth or medical compressed air. The handle may need to be disassembled to ensure thorough drying.

STERILIZATION

Use one (1) of the following four (4) sterilization methods to sterilize the SunMed® reusable GreenLine® conventional laryngoscope handles and blades.

STEAM STERILIZATION

Insert laryngoscope handle (without batteries for both waterproof and non-waterproof handles) or laryngoscope blades in appropriate autoclave pouch.

	(B) GRAVITY AUTOCLAVE	(C) PRE-VACUUM STEAM
Temperature	121° C (250° F)	134° C (273° F)
Cycle Time	30 Minutes	3 Minutes
Dry Time	15 Minutes	20 Minutes

(D) STERIS® AMSCO® V-PRO® STERILIZATION Conventional Laryngoscope Blades and Handles are compatible with:

- Amsco® V-PRO® 1 Low Temperature Sterilization System
- Amsco® V-PRO® 1 Plus Low Temperature Sterilization System
- Amsco® V-PRO® maX Low Temperature Sterilization System

(E) STERRAD® STERILIZATION Conventional Laryngoscope Blades and Handles are compatible with:

- STERRAD® 100NX® System (Standard and EXPRESS Cycle)
- STERRAD® NX® System (Standard Cycle)
- STERRAD® 100S and 200 System (Short cycle Outside US)
- STERRAD® 50 System

BATTERY / LAMP TESTING

Laryngoscope systems (laryngoscope blades and laryngoscope handles) must be tested after each cleaning/high-level disinfection or cleaning/sterilization, prior to use, and after battery replacement. Fully engage blade on the handle. If the lamp guide fails to light, check the batteries/lamp for correct placement or replace with new ones.

To Install Batteries: Place two (2) 1.5 -volt batteries, positive end first into Penlight, Medium, or Large handle (Fig 1) and reinstall the end cap. In Stubby handle, insert battery cartridge according to instructions marked on cartridge (Fig 2). If Stubby handle lamp fails to light, batteries may need to be reversed for correct polarity. Attach Conventional blade in operating position. Lamp should illuminate.

To Change Lamp: Prior to lamp removal, allow lamp time to cool. To replace the lamp, unscrew the lamp from the blade, and pull the lamp out. Reinstall a new lamp in the laryngoscope blade, and tighten it. Ensure the lamp is seated properly and verify lamp function prior to use.

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SunMed, LLC,
Fabricado para SunMed, LLC,
una subsidiaria de SunMed Holdings, LLC