

REF 0370

UNITS - 1
RX Only



CE 0482

OXYGEN MASK

Adult Oxygen Mask with 7' Tubing

CAUTION: Use of sterilization or cleaning solutions may leave this device nonfunctional. When using the mask with no vents, the clinician should always be present during the treatment.

SUGGESTED USE:

1. These products are intended for use in delivering oxygen therapy.
2. All products are to be used in accordance with the applicable prescription.
3. Attach the Mask to the components included in this kit, and then connect the supply tube to the gas source.
4. Ensure that all connections are secure, and then start gas flow.
5. Check for appropriate function prior to use.
6. Discard after single patient use.

Westmed

5580 S. Nogales Highway
Tucson, Arizona 85706 USA
Phone: 800-975-7987
Fax: 520-294-6061
www.westmedinc.com



MT Promedt Consulting GmbH
Altenhofstr. 80
66386 St. Ingbert, Germany
Patent Pending

Assembled in Mexico
LABEL PN 70264, Rev. 07



LOT