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REF 002620

AirLife™

en - Prefilled Humidifier. Sterile Water for Inhalation,
Nonpyrogenic pH ~ 5 • USP, 500 ml with Humidifier Adapter
Applicable for Canada only: Medicinal Ingredient: Water 1 g/ml

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Imported in Canada by
Cardinal Health Canada, Inc.
Mississauga, Ontario
L5T 1C2 Canada
SterilWater.INIFU Rev. 1

2024/11

Sterile Water for Inhalation, USP, 500 ml with Humidifier Adapter

	Catalogue number		U.S. Federal law restricts this device to sales by or on the order of a physician
	Lot Number		Manufacturer
	Quantity		Use by Date
	Product and packaging are not made with natural rubber latex.		For Single Use Only
	Caution		Sterilize Fluid path Dry Heat
	Consult instructions for use		Do not use if package is damaged
	Warning		Keep away from sunlight

Approximate fluid contents 500 ml. No bacteriostatic or other preservatives added. • Applicable for Canada Only: Medicinal Ingredient: Water 1 g/ml • Sterile, nonpyrogenic pH ~ 5. • Latex free • External surfaces of bottle not sterile. • Recommended storage: Room temperature (25°C or 77°F). Avoid excessive heat. Protect from freezing.

Directions for use:

1. Remove humidifier adapter from package. Remove break-away “trigger” from bottle top by sharply pulling up on “trigger.” Secure humidifier adapter to bottle by turning in a clockwise direction until adapter has been completely seated to the bottle top.
2. Attach the humidifier adapter nut to the outlet of the flow meter. Remove break-away “trigger” from outlet of humidifier with an upward snapping motion. Do not twist off.
3. Attach delivery tube to the outlet port of the humidifier.
4. To check audible relief alarm, set flow meter at 4 LPM, fold back and pinch tubing. When audible relief alarm is heard, release tubing and proceed with therapy.
5. Adjust liter flow in accordance with doctor’s prescription.

Warning: Read humidifier bottle label prior to use.

Caution: Do not use sterile water for inhalation for parenteral administration.

Contraindications: Do not use unless solution is clear and container intact.

Caution: For single use only. Re-use may degrade the performance of the product or contribute to cross contamination.

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