REF 5251

AirLife™

Modudose 0.9% Sodium Chloride Solution, USP, 3 ml

Applicable for Canada Only: Medicinal Ingredient: Sodium Chloride 0.90 g/100 ml

Non-pyrogenic pH ~ 5, preservative-free single-use vials for respiratory therapy.

No bacteriostatic agent or other preservative added.

NPN 80023553

	Catalogue number		U.S. Federal law restricts this device to sales by or on the order of a physician
	Lot Number		Manufacturer
	Quantity		Use by Date
	Product and packaging are not made with natural rubber latex.		For Single Use Only
	Caution		Sterilize Fluid path Aseptic
	Consult instructions for use		Do not use if package is damaged
	Warning		Keep away from sunlight

en - English

Indications:

Contents of these vials are for use in respiratory therapy and for tracheal lavage.

Warning:

- Not for injection or in preparations to be used for injection. Not for parenteral administration.

Directions for Use

1. To verify container integrity, squeeze the vial before use. Twist cap completely off vial, invert and squeeze prescribed volume.
2. Discard any unused portion of the contents of this single-use vial as well as any unused solution remaining in the nebulizer cup.

3. Contents are sterile in the unopened, undamaged package.

Recommended storage:

Room temperature (25 °C or 77 °F). • Avoid excessive heat. Protect from freezing.

Caution:

- For single use only. Re-use may degrade the performance of the product or contribute to cross contamination.
- **USA:** Federal (USA) law restricts this device to sale by or on the order of a physician or licensed practitioner.

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Cardinal Health Canada, Inc.