REF AL4109

AirLife™

Sterile 0.9% Sodium Chloride for Irrigation, USP, 110 ml

Non-pyrogenic pH ~ 5 • Applicable for Canada Only: Medicinal Ingredient: Sodium Chloride 0.90 g/100 ml • Recommended storage: Room temperature (25°C or 77°F). Avoid excessive heat. Protect from freezing.

REF	Catalogue number	Rx ONLY	U.S. Federal law restricts this device to sales by or on the order of a physician
LOT	Lot Number	Factory	Manufacturer
QTY	Quantity	Hourglass	Use by Date
Warning	Caution	No use if damaged	Do not use if package is damaged
Info	Consult instructions for use	Single use	For Single Use Only
Sterile	Sterilize Fluid path Dry Heat	Sunlight	Keep away from sunlight

en - English

Caution:

- For single use only. Re-use may degrade the performance of the product or contribute to cross contamination.

Directions For Use:

Use aseptic technique. To pour contents, remove cap by grasping at base of cap at bottleneck. Snap cap at 90° angle to open. To squirt contents, gently twist off the top cap by grasping the tab and turning. Do not squeeze bottle while opening. Discard after initial use. Do not store after opening. Sterility guaranteed in the unopened undamaged package.

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Cardinal Health Canada, Inc.