REF AL7023/AL7025

AirLife™

Modudose™ Sterile Water for Inhalation, USP, 3 ml/5 ml

Applicable for Canada Only: Medicinal Ingredient: Water 1 g/ml

Non-pyrogenic pH ~ 5, preservative-free single-use vials for respiratory therapy.

No bacteriostatic agent or other preservative added.

NPN 80023548

REF	Catalogue number	R _X ONLY	U.S. Federal law restricts this device to sales by or on the order of a physician
LOT	Lot Number		Manufacturer
QTY	Quantity		Use by Date
	Product and packaging are not made with natural rubber latex.		For Single Use Only
	Caution		Sterilize Fluid path Aseptic
	Consult instructions for use		Do not use if package is damaged
	Warning		Keep away from sunlight

en - English

Indications:

Contents of these vials are for use in respiratory therapy and for tracheal lavage.

Warning:

- Not for injection or in preparations to be used for injection. Not for parenteral administration.

Directions for Use

1. To verify container integrity, squeeze the vial before use. Twist cap completely off vial, invert and squeeze prescribed volume.
2. Discard any unused portion of the contents of this single-use vial as well as any unused solution remaining in the nebulizer cup.

AirLife
2710 Northridge Dr. NW, Suite A
Grand Rapids, MI 49544 USA
www.myAirLife.com

1079493
Imported in Canada by
Cardinal Health Canada, Inc.
Mississauga, ON
L5T 1C2 Canada
ModSter3-5ml.INIFU

Rev. 1 2024/11

3. Contents are sterile in the unopened, undamaged package.

Recommended storage:

Room temperature (25 °C or 77 °F). • Avoid excessive heat. Protect from freezing.

Caution:

- For single use only. Re-use may degrade the performance of the product or contribute to cross contamination.
- **USA:** Federal (USA) law restricts this device to sale by or on the order of a physician or licensed practitioner.

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