EN

English

Salter Labs™ Aerosol Face Tent

DEVICE DESCRIPTION:
The device is non-sterile, disposable, and for single-patient multi-use. The aerosol mask is a device placed over a patient's nose or mouth to administer aerosols.

INTENDED PURPOSE:
Aerosol Masks are intended to be used for aerosolized medication delivery applications.

INDICATIONS:
Aerosol mask is used in conjunction with a nebulizer to deliver aerosolized medication to the mouth and nose in spontaneously breathing patients.

ENVIRONMENT:
Hospitals, sub-acute, medical clinics, pre-hospital, home, surgical centers, skilled nursing facilities.

PATIENT TARGET GROUP:
Infant, pediatric, adult

INTENDED USER:
Medically trained personnel.

EXPECTED CLINICAL BENEFITS:
Alternate interface to deliver aerosolized medication for patients unable to use a mouthpiece.

CONTRAINDICATIONS:
None Known

⚠️ WARNINGS:

- If oxygen is in use, do not use near flame or heat source.
- Device contains components that may present choking hazard.
- Use a flow rate ≥ 5 LPM to prevent CO₂ accumulation and re-breathing.

⚠️ CAUTIONS:

- Clinician or caregiver should avoid inhaling aerosolized medication.
- Consult physician if patient develops infection, skin irritation, or material sensitization.

RESIDUAL RISKS:
Refer to Warnings and Cautions.

NECESSARY USER QUALIFICATIONS:
The end-user should receive training on the proper use of the device and be familiar with the indications for use, warnings and cautions as stated in the IFU.
RX Only: Federal law (USA) restricts this device to sale by or on the order of a licensed healthcare practitioner.

INSTRUCTIONS:
Place mask over nose and mouth, adjust elastic strap and nose bar for a snug fit. If connection is loose, use Salter Aerosol Mask Adapter. Ensure aerosol mist is visible. Follow instructions for proper care of device to prevent risk of contamination and injury. This product is disposable and is not intended for prolonged use. Check for leaks at connections and for proper placement prior to use.

CLEANING/REUSE INSTRUCTIONS
A discontinued use of this product on the same patient, with periods of use separated by a period of non-use, is acceptable. The user must ensure that the product is not damaged or contaminated between usages on the same patient.

SAFE DISPOSAL:
Dispose of all materials in accordance with local, state, and federal regulations. Decontaminate and dispose of all potentially biohazardous material.

INCIDENT REPORTING:
Notice to the user that any serious incident that has occurred in relation to the device should be reported to the manufacturer at, productquality@myairlife.com, and the competent authority of the Member State in which the user and/or patient is established.

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MT Promedt Consulting GmbH
MT Promedt Consulting Ltd.
MedEnvoy UK Limited