




JP 性能特性

品目	名称	サイズ	理想体重	理想体重	最大OETT	口の開き	チューブの長さ	最大OGチューブ
40005	Air-Qsp*3	0	< 4 lbs	< 2 kg	3.0 mm	5 mm	6.29 cm	—
40055	Air-Qsp*3	0.5	4 - 9 lbs	2 - 4 kg	4.0 mm	8 mm	7.78 cm	—
40105	Air-Qsp*3	1	9 - 15 lbs	4 - 7 kg	4.5 mm	11 mm	9.02 cm	—
40155	Air-Qsp*3	1.5	15 - 37 lbs	7 - 17 kg	5.0 mm	14 mm	~10.5 cm	—
40205	Air-Qsp*3	2	37 - 66 lbs	17 - 30 kg	5.5 mm	17 mm	11.63 cm	—
40305	Air-Qsp*3	3	66 - 132 lbs	30 - 60 kg	7.0 mm	20 mm	13.04 cm	—
40405	Air-Qsp*3	4	132 - 176 lbs	60 - 80 kg	8.0 mm	23 mm	14.99 cm	—
40505	Air-Qsp*3	5	> 176 lbs	> 80 kg	9.0 mm	25 mm	16.5 cm	—
60005	Air-Qsp*3G	0	< 4 lbs	< 2 kg	3.0 mm	5 mm	6.86 cm	5 Fr
60055	Air-Qsp*3G	0.5	4 - 9 lbs	2 - 4 kg	4.0 mm	8 mm	8.05 cm	6 Fr
60105	Air-Qsp*3G	1	9 - 15 lbs	4 - 7 kg	4.5 mm	11 mm	9.29 cm	8 Fr
60155	Air-Qsp*3G	1.5	15 - 37 lbs	7 - 17 kg	5.0 mm	14 mm	10.9 cm	8 Fr
60205	Air-Qsp*3G	2	37 - 66 lbs	17 - 30 kg	5.5 mm	17 mm	11.61 cm	10 Fr
60305	Air-Qsp*3G	3	66 - 132 lbs	30 - 60 kg	7.0 mm	20 mm	14.25 cm	14 Fr
60405	Air-Qsp*3G	4	132 - 176 lbs	60 - 80 kg	8.0 mm	23 mm	15.37 cm	16 Fr
60505	Air-Qsp*3G	5	> 176 lbs	> 80 kg	9.0 mm	25 mm	17.75 cm	18 Fr

DISPOSABLE AIR Q3® EXCHANGER

品目	スタイルットサイズ	AIR-Q®3/3Gサイズ	最大OETT	重量	バック
10 - 1006	00	0 - 1.5	4.5 mm	< 17 kg	10
10 - 1005	0	2 - 3	6.0 mm	17 - 60 kg	10
10 - 1004	1	4 - 5	9.0 mm	> 60 kg	10


記号用語集	
	チューブの長さ
	理想体重

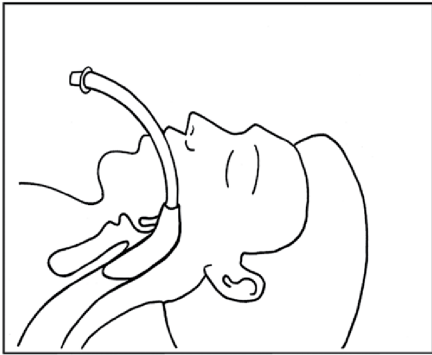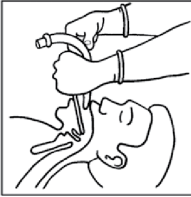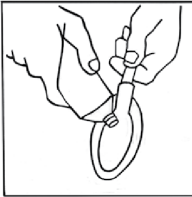


RECOMMENDED INSERTION TECHNIQUE
FR: TECHNIQUE D'INSERTION RECOMMANDÉE
ES: TÉCNICA DE INSERCIÓN RECOMENDADA
IT: TECNICA DI INSERIMENTO CONSIGLIATA
DE: EMPFOHLENE INSERTIONSTECHNIK
EL: ΕΥΝΙΣΤΟΜΕΝΗ ΤΕΧΝΙΚΗ ΕΙΣΑΓΩΓΗΣ
PT: TÉCNICA DE INSRÇÃO RECOMENDADA
PL: ZALECANA TECHNIKA WPROWADZANIA
NL: AANBEVOLEN INBRENGTECHNIEK
DA: ANBEFALET INDFØRINGSTEKNIK
SV: REKOMMENDERAD INFÖRINGSTEKNIK
NO: ANBEFALT INNFORINGSTEKNIKK
FI: SUOSITELTU SISÄÄNVIENTITEKNIikka
TR: ÖNERİLEN YERLEŞTİRME TEKNİĞİ
RU: РЕКОМЕНДУЕМАЯ МЕТОДИКА ВВЕДЕНИЯ
ZH: 建议的插入技术

RECOMMENDED DEPTH OF INSERTION RANGE
FR: PROFONDEUR D'INSERTION RECOMMANDÉE
ES: INTERVALO DE PROFUNDIDAD DE INSERCIÓN RECOMENDADO
IT: PROFONDITÀ DI INSERIMENTO CONSIGLIATA
DE: EMPFOHLENER INSERTIONSTIEFENBEREICH
EL: ΕΥΝΙΣΤΟΜΕΝΟ ΒΑΘΟΣ ΕΥΡΟΥΣ ΕΙΣΑΓΩΓΗΣ
PT: PROFUNDIDADE DE INSRÇÃO RECOMENDADA
PL: ZAKRES ZALECANEJ GŁĘBOKOŚCI WPROWADZENIA
NL: AANBEVOLEN BEREIK INBRENGDIEPTE
DA: REOMMENDED DYBDE AF INDFØRINGSOMRÅDE
SV: INTERVALL FÖR REKOMMENDERAT INFÖRINGSDJUP
NO: ANBEFALT INNFORINGSDYBDE
FI: SUOSITELTU SISÄÄNVIENTISYVYYS
TR: ÖNERİLEN YERLEŞTİRME DERİNLİĞİ ARALIĞI
RU: РЕКОМЕНДУЕМЫЙ ДИАПАЗОН ГЛУБИНЫ ВВЕДЕНИЯ
ZH: 建议的插入深度范围

LEGEND
Incisors
Tongue
Hyoid Bone
Vocal Cords/Folds
Thyroid Cartilage
Trachea
Esophagus
Laryngeal Inlet
Epiglottis
Arytenoid Cartilages
Sealing Mechanism
Venilatory Opening
Recommended Depth of Insertion Marks
Connector
Cervical Vertebrae #2
Oral Pharynx
Nasal Pharynx
Mandible

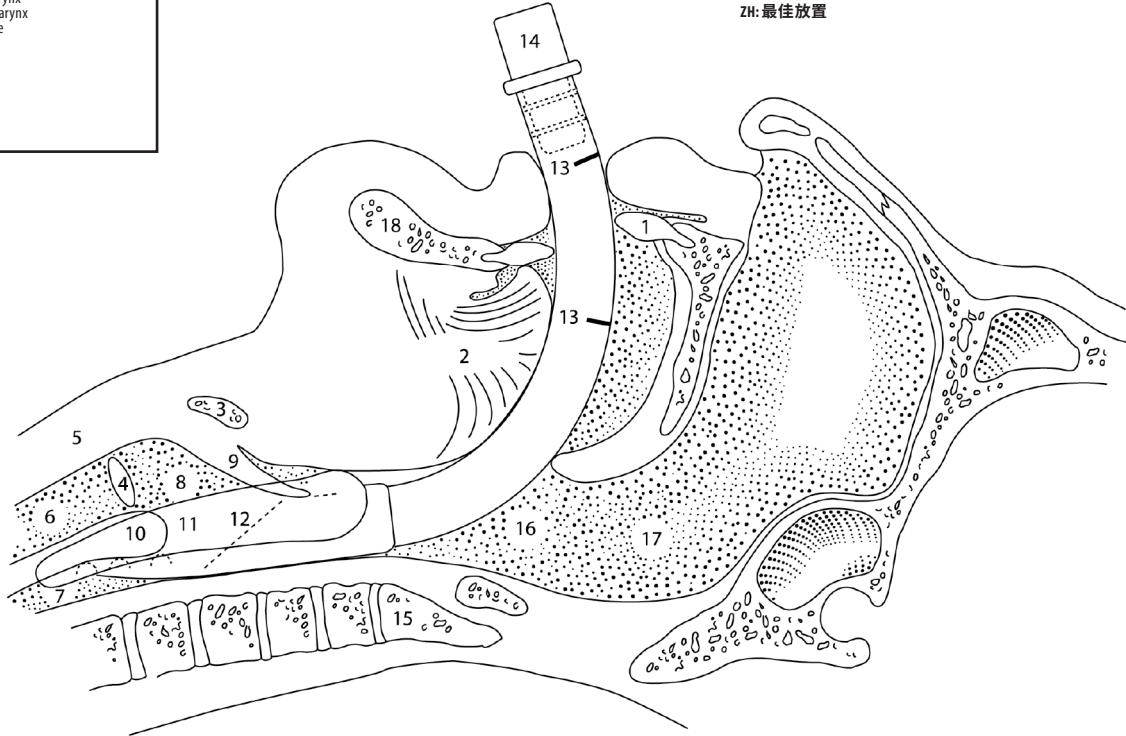

OPTIMAL PLACEMENT
FR: MISE EN PLACE OPTIMALE
ES: COLOCACIÓN ÓPTIMA
IT: POSIZIONAMENTO OTTIMALE
DE: OPTIMALE PLATZIERUNG
EL: ΒΕΛΤΙΣΤΗ ΤΟΠΟΘΕΤΗΣΗ
PT: COLOCAÇÃO IDEAL
PL: OPTYMALNE UMIESZCZENIE
NL: OPTIMALE PLAATSING
DA: OPTIMAL PLACERING
SV: OPTIMAL PLACERING
NO: OPTIMAL PLASSERING
FI: OPTIMAALINEN ASETTAMINEN
TR: OPTİMUM YERLEŞTİRME
RU: ОПТИМАЛЬНАЯ УСТАНОВКА
ZH: 最佳放置

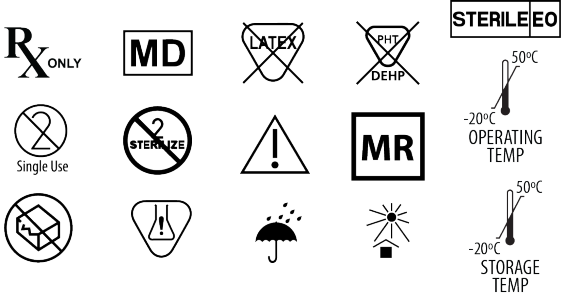

SunMed, LLC.
2710 Northridge Dr. NW, Suite A
Grand Rapids, MI 49544 USA
www.myAirLife.com
Made in China

UK REP
MT Promedt Consulting Ltd.
Beaver House, 23-38
Hythe Bridge Street
Oxford, OX1 2EP
United Kingdom

AU REP
Australian Sponsor:
AirLife Australia Holdings Pty Ltd
PO Box 97
North Ryde BC, NSW, 1670
Australia

EC REP
MT Promedt Consulting GmbH
Ernst-Heckel-Straße 7
66386 St. Ingbert
Germany

MedEnvoy UK Limited
85, Great Portland Street – First Floor
London, W1W 7LT
United Kingdom

MedEnvoy
NL-IM-00000248
Prinses Margrietplantsoen 33
Suite 123
2595 AM, The Hague
The Netherlands

CH REP
MedEnvoy Switzerland
Gotthardsstrasse 28
6302 Zug
Switzerland

