Hybrid HV Face Mask

Instructions for Use – 10039 Rev C

Intended Use

The Hybrid HV Face Mask is intended to provide a patient interface for application of noninvasive ventilation. This mask is to be used as an accessory to ventilators which have adequate alarms and safety systems for ventilator failure and which are intended to administer positive pressure ventilation. A separate exhalation device and safety valve must be used with this mask.

The Hybrid HV is intended for use on adult patients (>30 kg), who are appropriate candidates for noninvasive ventilation. The Hybrid HV may be reused multiple times by a single patient.

Specifications


- The deadspace associated with this mask is less than 145 cm³.
- The maximum unintended leak from parts of the mask is 3.5 LPM at 20 cm H₂O and 5.0 LPM at 35 H₂O.
- Storage Temperature: -20 to 60°C (-4 to 140°F), Operational Temperature: 5 to 40°C (-41 to 104°F), Humidity: 0–95%

Warnings

- In the USA, federal law restricts this mask to sale by, or on the order of, a physician. (RX Only)
- This mask does not include an anti-asphyxia valve that would allow the patient to breathe if the ventilator failed. The mask should only be worn when the ventilator is turned on and operating properly. This mask must be used with ventilators which have adequate alarms and safety systems for ventilator failure. In the event of ventilator failure, use of this mask requires the same level of attention and assistance as the use of a tracheal tube.
- If the patient experiences skin irritation, consult the physician. The mask consists of silicone, polycarbonate, polypropylene, nylon, polyester, and neoprene.
- Contact the physician if the patient experiences unusual chest discomfort, shortness of breath, stomach distention, belching, or severe headache when he/she awakens or when he/she is receiving positive airway pressure.
- To minimize the risk of vomiting during sleep, the patient should avoid eating or drinking for three hours before using the mask. This mask is not recommended if the patient is taking a prescription drug that may cause vomiting.
- This mask is intended for single-patient use only and thus use by multiple patients is to be avoided due to the risk of transmitting infection. Do not attempt to sterilize or use disinfectants on this mask.


Contraindications

This mask is contraindicated for patients without a spontaneous respiratory drive. This mask may not be suitable for persons with the following conditions: impaired cardiac sphincter function, excessive reflux, impaired cough reflex, and hiatal hernia. It should not be used if patient is uncooperative, obtunded, unresponsive, or unable to remove the mask.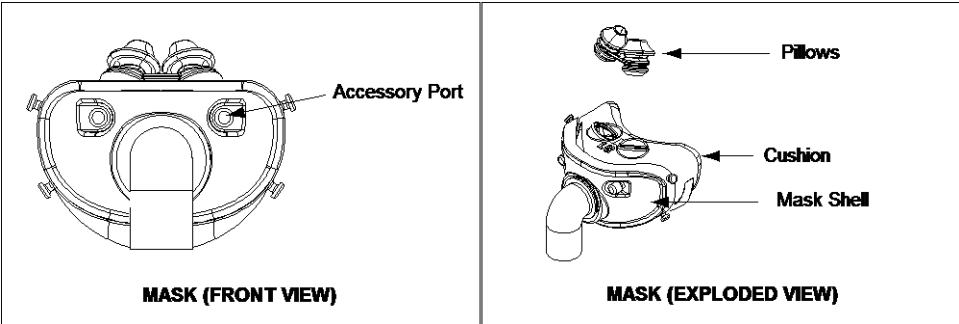

Instructions for Use

- Prior to use, clean the mask as described in the cleaning instructions.
- Clean and thoroughly dry the patient's face before putting on the mask.
- Carefully inspect each component for damage. If you see any cracks or tears, do not use the mask.
- Insert the nasal pillows into the cushion. NOTE: Pillows can be attached by their lower groove (Tallest Height Position) or their upper groove (Shortest Height Position) in order to optimize your seal.
- Attach the two upper clips of the headgear to the two upper pegs on the mask and pull the clips back (towards the face) until an audible "click" is heard. NOTE: The "loop" surface on the headgear should face outward and the black buckle should be at the top.
- Attach the circuit tube from the ventilation system to the elbow. *NOTE: The Hybrid NE is designed to accept a 22 mm male connector. If the hose has a female connector, a male-male adaptor can be used between the elbow and hose.
- Insert the pillows into the nose and then rotate the mask down over the mouth. NOTE: The upper wall of the cushion should sit on the upper lip, not on the mouth.
- Pull the headgear back over the top of the head. Attach the two lower clips of the headgear to the two lower pegs on the mask and pull the clips back (towards the face) until an audible "click" is heard.

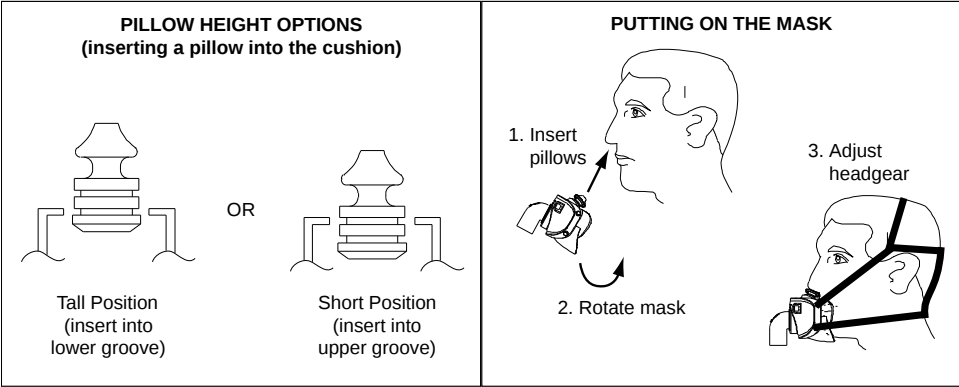Adjusting and Removing the Mask

- Pull the back of the headgear up on the head until the top side straps of the headgear are well above the ears.
- Gently pull each of the four side straps to achieve an overall even, balanced pressure and fit between the cushion and the face. NOTE: Over-tightening may cause leaks.
- Gently pull each of the two Over-Head straps until the pillows fit comfortably against the nose. NOTE: If you are unable to achieve a comfortable fit or to resolve leaks at the pillows, try the following:
 - the pillows in a different Height Position
 - an alternative pillow size
- To remove the mask, gently press the mask towards the face and push the lower two clips away from face until they unsnap. Lift the mask off over the head.

Cleaning Instructions

After Each Use:

- Remove the headgear from the mask as the headgear will be washed separately. The headgear should be handwashed following the remaining instructions at least every 7 days. Do not machine wash.
- Disassemble the pillows from the mask.
- Handwash all components in warm, soapy water. NOTE: Only use pure soap and do not use alcohol, petroleum products, or strong cleaners as these may cause damage and reduce the life of the product.
- Rinse thoroughly and shake excess water from interior surfaces of components. Allow all components to completely air-dry, away from direct sunlight.
- Carefully inspect each component for any wear or damage. Discontinue use if you notice any cracks in the plastic, any tears or hardening of the silicone, or significant fraying or loss of elasticity of the headgear, and contact your healthcare provider for a replacement. Discoloration is normal after several washings and should not be considered sign of wear.

Manufactured for:

30 Spur Dr., El Paso – TX 79906 – USA
Tel: (800) 421-0024 – www.salterlabs.com

*Covered by Patents: U.S.D550352, U.S.D550836, U.S.D557801, U.S.D564089, U.S.D564090, U.S.D568985, U.S.D583047, U.S.D583048, U.S.D586910, U.S.D597659, U.S.8042539, U.S.8261745, U.S.8752551, U.S.8887725.
Other patents pending.*

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