**HYBRID®**

EN: ENGLISH

**Hybrid Face Mask**


Instructions for Use – 05080 Rev K

**Intended Use**

The Hybrid is intended for use by adults (> 30 kg) as a patient interface for CPAP or Bi-Level positive-pressure ventilation devices for treatment of respiratory insufficiencies and obstructive sleep apnea. The Hybrid may be reused multiple times by a single patient.

Storage Temperature:  
-20° to 60°C (-4° to 140°F)  
Humidity: 0–95%

The deadspace associated with this device is less than 125 cm<sup>3</sup>.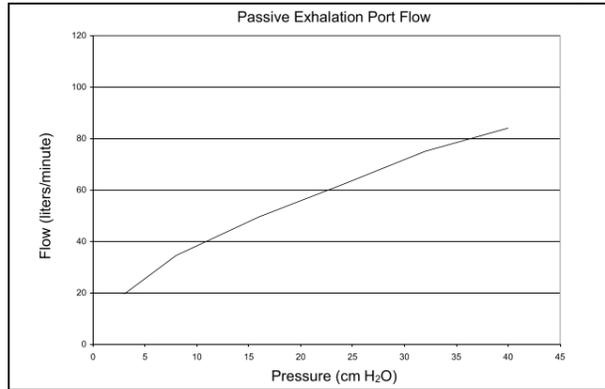

**Warnings**

- In the USA, federal law restricts this mask to sale by, or on the order of, a physician. (RX Only)
- This mask should be used only with CPAP or Bi-level systems recommended by your physician or respiratory therapist. A mask should not be used unless the CPAP or Bi-level system is turned on and operating properly. The exhalation ports should never be blocked. *Explanation of the Warning:* CPAP/Bi-level systems are intended to be used with special masks which have vent holes or exhalation ports to allow continuous flow of air out of the mask. When the CPAP/Bi-level machine is turned on and functioning properly, new air from the machine flushes the exhaled air out through the attached mask exhalation ports. However, when the CPAP/Bi-level machine is not operating, enough fresh air will not be provided through the mask, and exhaled air may be rebreathed. Rebreathing of exhaled air for longer than several minutes can in some circumstances lead to suffocation. This warning applies to most models of CPAP and Bi-level systems.
- At low CPAP pressures, the flow through the exhalation ports may be inadequate to clear all exhaled gas from the tubing. Some rebreathing may occur. A minimum pressure of 3.0 cm H<sub>2</sub>O at the mask is required.
- The anti-asphyxia valve allows the patient to breathe room air in the case that the CPAP/Bi-level device connected to the mask is not operating or is turned "off". The mask should not be used if the anti-asphyxia valve is missing, damaged, or not working properly. The large opening at the valve should never be blocked.
- Do not use the mask if vomiting or feeling nauseous. Failure to arouse and remove the mask after vomiting could result in aspiration of vomitus. To minimize the risk of vomiting during sleep, avoid eating or drinking for three hours prior to sleep. This mask is not recommended if the patient is taking a prescription drug that may cause vomiting.
- This device is intended for single-patient use only and thus use by multiple patients is to be avoided due to the risk of transmitting infection. Do not attempt to sterilize or use disinfectants on this device or any of its components.
- The mask may deactivate the pressure alarm on some ventilation systems. Confirm that your ventilation system pressure alarm works with the product before using the pressure alarm.
- If you experience any reaction or notice redness or irritation of your skin, contact your physician. The cushions and pillows consist of silicone. Other mask components consist of silicone, polycarbonate, polypropylene, nylon, polyester, and neoprene.
- Any unusual chest discomfort, shortness of breath, stomach distention, belching or severe headache during or immediately after use should be immediately reported to your physician.
- If oxygen is used with the CPAP or Bi-level device, the oxygen flow must be turned off when the flow generator is not operating. *Explanation of the warning:* When the CPAP or Bi-level device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the flow generator enclosure. Oxygen accumulated in the CPAP or Bi-level device enclosure will create a risk of fire. This warning applies to most types of CPAP or Bi-level flow generators. Do not smoke while oxygen is in use.  
Note: At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection, and the leak rate. This warning applies to most types of CPAP systems.

**Contraindications**

This mask may not be suitable for persons with these conditions: impaired cardiac sphincter function, excessive reflux, impaired cough reflex, and hiatal hernia. It should not be used if patient is uncooperative, obtunded, unresponsive, or unable to remove the mask.

**Instructions for Use**


- Prior to use, clean the mask as described in the cleaning instructions.
- Clean and thoroughly dry the patient's face before putting on the mask.
- Attach the swivel coupling to the mask elbow.
- Insert the nasal pillows into the cushion. NOTE: Pillows can be attached by their lower groove (Tallest Height Position) or their upper groove (Shortest Height Position) in order to optimize your seal.
- Insert the cushion into the groove on the mask shell.
- Inspect the anti-asphyxia valve located within the clear elbow. With the CPAP or Bi-level system turned off, verify that the valve is positioned so that room air can flow through the large opening near the valve in the elbow. Turn on the system to 3 cmH<sub>2</sub>O; the valve should close the large opening and system air should flow into the mask. If the valve does not close or does not function properly, replace the mask.
- Attach the two upper clips of the headgear to the two upper pegs on the mask and pull the clips back (towards the face) until you hear a "click". NOTE: The "loop" surface on the headgear should face outward and the black buckle should be at the top.
- Insert the pillows into the nose and then rotate the mask down over the mouth. NOTE: The upper wall of the cushion should sit on the upper lip, not on the mouth.
- Pull the headgear back over the top of the head. Attach the two lower clips of the headgear to the two lower pegs on the mask and pull the clips back (towards the face) until an audible "click" is heard.
- Turn on CPAP or Bi-level system and make sure it is functioning properly. Connect the mask.

**Adjusting and Removing the Mask**

- Unhook the hook-and-loop closures of each of the five headgear straps individually and gently pull until the mask sits squarely on your face. The mask should fit snug enough to prevent large leaks yet still be comfortable. In general, a looser fit is better. Over-tightening may actually make leaks worse.
- Breathe normally, and continue making minor adjustments until you have a comfortable fit with minimal leaks. If you are unable to achieve a comfortable fit or to resolve any leaks around the mouth, you may need an alternative cushion size. If you are unable to achieve a comfortable fit or to resolve any leaks at the nose, you may need an alternative pillow size.
- To remove the mask, gently press the mask towards the face and push the lower two clips away from face until they unsnap. Lift the mask off over the head.

**Cleaning Instructions (After Each Use):**

- Remove the headgear from the mask; it will be washed separately. The headgear should be handwashed following the remaining instructions at least every 7 days. Do not machine wash.
- Disassemble swivel coupling and cushion from the mask. Remove pillows from the cushion.
- Wash the swivel coupling, cushion, pillows, and the remaining mask in warm water with a mild dishwashing detergent. Do not use alcohol or petroleum products or strong cleaners.
- Make sure that the exhalation ports are free of any obstructions. Use a toothpick or similar item to remove any debris.
- Rinse thoroughly. Allow all components to completely air-dry, away from direct sunlight.
- Carefully inspect each component for any wear or damage. Discontinue use if you notice any cracks in the plastic, any tears or hardening of the silicone, or significant fraying or loss of elasticity of the headgear, and contact your healthcare provider for a replacement. Discoloration is normal after several washings and should not be considered sign of wear.

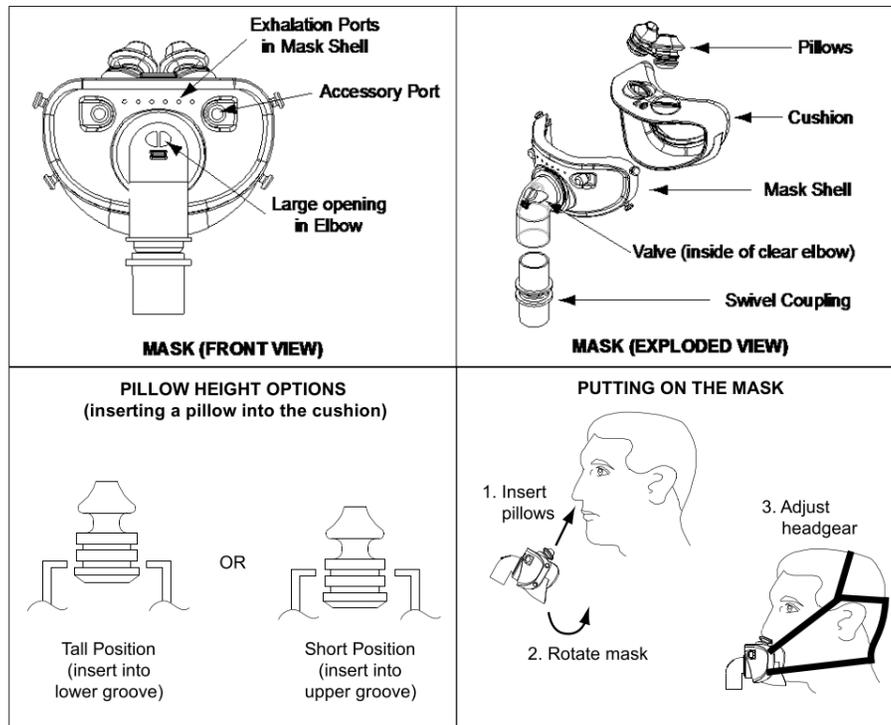**Manufactured for:** **SALTER LABS®** 30 Spur Dr., El Paso – TX 79906 – USA  
Tel: (800) 421-0024 – www.salterlabs.com

\*Covered by Patents: U.S. D550352, U.S. D550836, U.S. D557801, U.S. D564089, U.S. D564090, U.S. D568985, U.S. D583047, U.S. D583048, U.S. D583049, U.S. D586910, U.S. D597659, U.S. 8042539, U.S. 8261745, U.S. 8752551, U.S. 8887725. Other patents pending. Hybrid logo is a U.S. registered trademark.

Reorder Information			
Part Number	Description	Part Number	Description
HYB500	Hybrid Face Mask System – All Sizes Kit	HYB521	Pillows, <i>Small</i>
HYB511	Cushion, <i>Small</i>	HYB523	Pillows, <i>Medium</i>
HYB513	Cushion, <i>Medium</i>	HYB525	Pillows, <i>Large</i>
HYB515	Cushion, <i>Large</i>	HYB530	Headgear

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