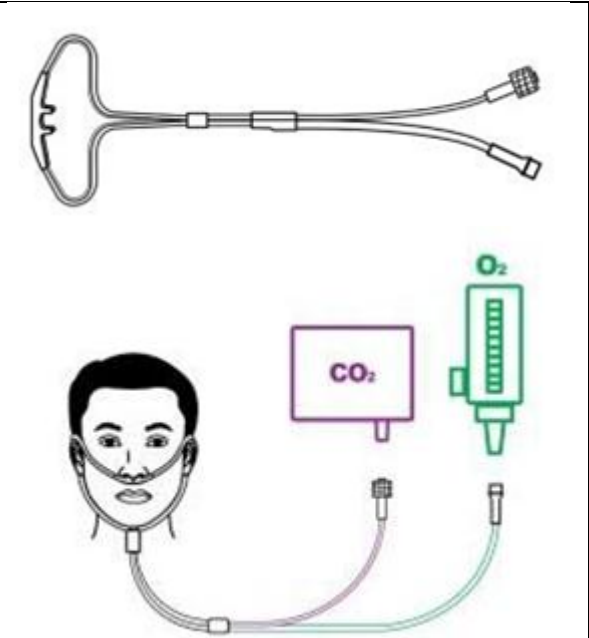
EN
DEVICE DESCRIPTION: EtCO₂/O₂ Nasal Sampling Cannula The device is a non-sterile, disposable, single patient use cannula that provides supplemental oxygen while sampling patients exhaled gases.
INTENDED PURPOSE: The O₂ Delivery / CO₂ Divided Nasal Cannula is intended to be used where low flow oxygen is administered and exhaled gas is monitored. **INDICATIONS:** The O₂ Delivery / CO₂ Sampling Nasal Cannula is intended to be used with spontaneoulsy breathing patients who require supplemental oxygen and exhaled gas monitored.
ENVIRONMENT: Hospital, sub-acute, pre-hospital
PATIENT TARGET GROUP: Adult Shelf life: 3 years
EXPECTED CLINICAL BENEFITS: 1) Improve oxygen saturation 2)Notifies clinicians of change in patient's ventilator status 3) Improves accuracy of ETCO₂ value while delivering oxygen flow
CONTRAINDICATIONS: None known **WARNINGS:** If oxygen is in use, do not use near flame or heat source Position tubing to avoid strangulation Patient may become hypoxic if oxygen flow is interrupted.
CAUTIONS: Consult physician if patient develops infection, skin irritation or material sensitization. Do not place anything on supply tubing that may obstruct flow Excessive patient secretion or a build-up of liquids in the tubing may occlude the sampling and require more frequent replacement. Not for use on patients with obstruction of both nostrils.
INSTRUCTIONS: Contact your country's Competent Authority to report any serious incident Dispose of device in accordance with local, state or national regulations Follow instructions for proper care of device to prevent risk of contamination and injury. This product is disposable and is not intended for prolonged use Check for leaks at connections and for proper placement prior to use Check for proper position to ensure detection of exhaled gas. Replace sampling line when blockage is indicated by the capnography monitor.

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