REF 001362

AirLife™

Adult Oxygen Mask

Vinyl, Under-the-Chin Style, 3 in 1 Mask with 7 feet (2.1 m) Crush Resistant Oxygen Tubing and U/Connect-It Adapter • For Single Patient Use

REF	Catalogue number	Rx ONLY	U.S. Federal law restricts this device to sales by or on the order of a physician
LOT	Lot Number	Clean, Ready to Use	Clean, Ready to Use
QTY	Quantity	Manufacturer	Manufacturer
	Not made with natural rubber latex	Expiration Date	Expiration Date
PHT DEHP	Contains Phthalates	Caution	Caution
	Consult instructions for use	Temperature Limit	Temperature Limit

en English

Intended Use

Intended for oxygen to be delivered to the patient at a prescribed concentration and/or flow rate. Oxygen tubing is connected to an oxygen source on one end and a mask on the other end to deliver oxygen to the patient.

- Nonrebreather
- Rebreather
- Simple

Directions for Use

Mask is assembled as a high concentration, nonrebreather mask with safety vent.

1. Attach oxygen tubing directly to threaded oxygen outlet. (Separate tubing adapter may be used but is not required.) Push and twist to seal.
2. Adjust oxygen flow as prescribed by physician.
3. Place mask over patient's face, covering nose and mouth.
4. Pinch nose clip to provide a seal around the nose.

AU REP
AirLife Australia Holdings Pty Ltd
PO Box 97
North Ryde BC, NSW, 1670
Australia

AirLife,
2710 Northridge Dr. NW, Suite A
Grand Rapids, MI 49544 USA
www.myAirLife.com
Made in Mexico

AirLife®

To use as a rebreather mask

Remove flexible discs located on side port and inside of mask on top of reservoir bag. Proceed with steps 1–4.

To use as a simple, medium concentration mask

Remove reservoir bag, connector with baffle, and flexible disc on side port. Insert oxygen connector (supplied loose in bag) and reconnect tubing. Proceed with steps 1–4.

Caution

When using mask with reservoir bag, make sure bag is inflated before attaching to patient.

Warning

- When used as a high concentration mask, assure adequate and uninterrupted oxygen flow to patient. If oxygen flow is inadequate or interrupted, patient may not be able to inhale, or may not be able to inhale adequate volume.
- The oxygen connector in mask should not be connected directly to any endotracheal or tracheostomy tube inlet.

Storage

Recommended normal storage temperature 0°C/+32°F to +50°C/+122°F.

Contains Phthalates (DEHP). Risks and Precautionary Measures related to Phthalates: This instruction pertains to the phthalate symbol marked on the device or its packaging. If this device is used for the treatment of children, or treatment of pregnant or nursing women; please note that the following types of procedures may increase the risk of exposure to phthalates: Exchange transfusion in neonates, total parenteral nutrition in neonates, multiple procedures in sick neonates, haemodialysis in peripubertal males, male foetus and male infant of pregnant women, and lactating women; and massive blood infusion into trauma patients. Although these procedures have the potential for increased risk of exposure, conclusive evidence of human health risks has not been established. As a precautionary measure, to reduce the potential for unnecessary exposures to phthalates, the product must be used in accordance with the instructions for use, and practitioners should refrain from using this product beyond the period of time the product is medically necessary or needed.

Dispose of all materials in accordance with local, state, and federal regulations. Decontaminate and dispose of all potentially biohazardous material.

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