AirLife™ Corrugated Flexible Polyethylene and EVA Tubing For Aerosol and IPPB Use • Segmented, 100 feet (30.5 m) Length, 22 mm I.D. Flat Pack

REF Catalogue number	R_X ONLY U.S. Federal law restricts this device to sales by or on the order of a physician
LOT Lot Number	Clean, Ready to Use Clean, Ready to Use
QTY Quantity	Consult Instructions for Use Consult Instructions for Use
LATEX Not made with natural rubber latex	DEHP Not made with DEHP
Caution	Date of manufacture
Manufacturer	For Single Patient Use

EN

English

Intended Use

Intended for oxygen to be delivered to the patient at a prescribed concentration and/or flow rate. Oxygen tubing is connected to an oxygen source on one end and a patient interface on the other end to deliver oxygen to the patient.

Directions for Use

Tubing has segmented cuffs every 6 inch (15 cm) and may be cut to desired length. Connect tubing to any standard 22 mm O.D. fixture or adapter in accordance with directions for use of corresponding equipment.

Caution

Do not use tubing where gas temperature exceeds 154°F/68°C.

Warning

Oxygen, while not flammable, supports combustion. Do not use near excessive heat flames, or other sources of ignition.

Dispose of all materials in accordance with local, state, and federal regulations. Decontaminate and dispose of all potentially biohazardous material.

AirLife
2710 Northridge Drive NW, Suite A
Grand Rapids, MI 49544 USA
www.myAirLife.com
Made in Mexico

AirLife Australia Holdings Pty Ltd.
PO Box 97
North Ryde BC, NSW 1670
Australia

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