REF 002058

AirLife™

Tee Adapter with Valve

15 mm O.D. x 15 mm I.D. ISO Connections

REF Catalogue number **R_X ONLY** U.S. Federal law restricts this device to sales by or on the order of a physician

LOT Lot Number **Clean, Ready to Use** Clean, Ready to Use

QTY Quantity **Manufacturer** Manufacturer

LATEX Not made with natural rubber latex **Date of Manufacture** Date of Manufacture

PHT/DEHP Not made with DEHP **Single Use** For single use

i Consult instructions for use

en

English

Intended Use

The valved tee adapter is intended for use with small volume nebulizers to deliver aerosolized medication for inhalation into a ventilator circuit.

Directions for Use

1. Place tee adapter in desired location in the circuit. Ensure a secure fit. Position the nebulizer port so that the tee adapter will be oriented vertically with the capped port pointed downward.
2. Maintain cap in closed position when not in use.
3. Open cap prior to using.
4. Activate the valve by placing the nebulizer into the base of the tee adapter. To ensure a tight fit, press and twist the nebulizer firmly into place.
5. Once procedure is completed, twist and pull to remove the nebulizer.
6. Reclose cap.

Dispose of all materials in accordance with local, state, and federal regulations. Decontaminate and dispose of all potentially biohazardous material.

AirLife
2710 Northridge Dr. NW,
Suite A
Grand Rapids, MI 49544 USA
www.myAirLife.com

AU REP AirLife Australia Holdings Pty Ltd
PO Box 97
North Ryde BC, NSW, 1670
Australia

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