REF 002060

AirLife™

Tee Adapter with Valve

22 mm O.D./O.D. ISO Connections

REF	Catalogue number		U.S. Federal law restricts this device to sales by or on the order of a physician
LOT	Lot Number		Clean, Ready to Use
QTY	Quantity		Manufacturer
	Not made with natural rubber latex		Date of Manufacture
	Not made with DEHP		For single use only
	Consult instructions for use		Caution

en

English

Intended Use

The valved tee adapter is intended for use with small volume nebulizers to deliver aerosolized medication for inhalation into a ventilator circuit.

Directions for Use

1. Place tee adapter in desired location in corrugated tubing.
2. Maintain cap in closed position when not in use.
3. Open cap prior to using.
4. Activate the valve by placing the nebulizer into the base of the tee adapter. To ensure a tight fit, press and twist the nebulizer firmly into place.
5. Once procedure is completed, twist and pull to remove the nebulizer.
6. Reclose cap.

Caution: For single use only. Re-use may degrade the performance of the product or contribute to cross contamination.

Dispose of all materials in accordance with local, state, and federal regulations. Decontaminate and dispose of all potentially biohazardous material.

 AirLife
2710 Northridge Dr. NW, Suite A
Grand Rapids, MI 49544 USA
www.myAirLife.com

 AirLife Australia Holdings Pty Ltd
PO Box 97
North Ryde BC, NSW, 1670
Australia

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