


ENFLOW® COMPETENCY CHECKLIST

NAME _____ DATE _____

FACILITY / HOSPITAL _____ DEPARTMENT _____

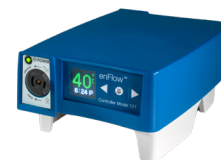

These assessment criteria are competency based. However, completion of the course in a non-clinical environment does not imply competence on the device in a clinical setting. The criteria is intended to constitute part of a continuing education program in the clinical and non-clinical environment, assisting in working towards competence but do not, in themselves, constitute competence. This document may assist in the development and self-assessment of competence or form part of an assessment by a designated hospital/trust trainer/educator. Assessment Statement for self-assessment/assessment by a designated hospital/trust trainer/educator: On completion of this assessment, I/the participant will be able to demonstrate understanding and both theoretical and clinical knowledge regarding the safety aspects and practical use of the enFlow® Warming System.

SKILLS CHECKLIST	COMPLETE	N/A
Identify and connect controller and warmer		
Connect power supply		
Connect cartridge and extension line to the IV fluid giving set		
Prime line using standard medically approved protocols ensuring cartridge is in an upward position (following direction of arrow on the cassette) while priming to eliminate any air in the line		
Turn device on		
Describe self-test		
Open the covers on the warmer and insert cassette		
Close cassette and describe self-test		
Place IV Line into IV Line clip to prevent kinking		
Connect to patient and commence infusion		
Describe the screen messages which will appear for the following scenarios:		
Warmer connected, power engaged, cartridge inserted and doors closed		
Warmer not connected but controller is powered on		
Warmer is connected but covers are not closed		
Warmer is connected, controller is powered on, covers are closed, cassette is not inserted into warmer		
Warmer over temperature		
Mute button activated		
Fault detected		
Describe the meanings of following LED colours:		
Solid Blue/Flashing Blue		
Solid Yellow/Flashing Yellow		
Solid Green/Flashing Green		
Red 1/2 second/Flashing Red		

SIGNATURE OF ASSESSOR _____

SIGNATURE OF TRAINEE _____