Vital Signs™ Lite Anesthesia Face Mask

REF Catalogue number

Rx ONLY U.S. Federal law restricts this device to sales by or on the order of a physician

LOT Lot Number

Manufacturer

Date of Manufacture

Not Made with DEHP

QTY Quantity

Caution

Not made with natural rubber latex

Exp Date

Consult Instructions for Use

Single Use

SIZE Mask Size

Size	Patient Population
1	Neonate
2	Infant
3	Toddler
4	Child
5	Small Adult
6	Medium Adult
7	Large Adult

EN

English

This product is not sterile. It is made of standard medical device materials and is a single patient disposable product. This product should not be used if physical deterioration, brittleness or discoloration occurs. Good clinical practice requires proper stock rotation to prevent older product from being stored for an excessive period of time.

Face mask with adjustable air cushion

Directions for use

1. Insert a luer tipped syringe into the valve and inflate/deflate mask cushion.

Placement of Mask:

1. Place mask cushion on the patient's chin, roll cushion up the face with a gentle but firm pressure to create an effective mask seal.
2. Refrain from pressing rigid part of the mask against the face, nose or eyes.
3. A head strap can be affixed to the optional prong ring or detached when not needed.

Distributed by

AirLife
2710 Northridge Dr. NW, Suite A
Grand Rapids, MI 49544 USA
www.myAirLife.com

AU REP AirLife Australia Holdings Pty Ltd
PO Box 97
North Ryde BC, NSW, 1670
Australia

AirLife®

WARNING

Do not use in the presence of flammable anaesthetics.
Do not rinse, soak, wash or sterilize.

CAUTION

Excessive pressure applied to face mask can lead to facial and/or optic nerve damage.

Single Use Devices:

This single-use product is not designed or validated to be reused. Reuse may cause a risk of cross-contamination, affect the measurement accuracy, system performance, or cause a malfunction as a result of the product being physically damaged due to cleaning, disinfection, re-sterilization, or reuse.

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