REF 001220

AirLife™

Face Tent Mask

Vinyl • For Single Patient Use

REF	Catalogue number	Rx ONLY	U.S. Federal law restricts this device to sales by or on the order of a physician
LOT	Lot Number	Clean, Ready to Use	Clean, Ready to Use
QTY	Quantity	Manufacturer	Manufacturer
LATEX	Product and packaging are not made with natural rubber latex	Expiration Date	Expiration Date
PHT DEHP	Contains Phthalates	Temperature Limit	Temperature Limit
i	Consult instructions for use		

pertains to the phthalate symbol marked on the device or its packaging. If this device is used for the treatment of children, or treatment of pregnant or nursing women; please note that the following types of procedures may increase the risk of exposure to phthalates: Exchange transfusion in neonates, total parenteral nutrition in neonates, multiple procedures in sick neonates, haemodialysis in peripubertal males, male foetus and male infant of pregnant women, and lactating women; and massive blood infusion into trauma patients. Although these procedures have the potential for increased risk of exposure, conclusive evidence of human health risks has not been established. As a precautionary measure, to reduce the potential for unnecessary exposures to phthalates, the product must be used in accordance with the instructions for use, and practitioners should refrain from using this product beyond the period of time the product is medically necessary or needed.

Dispose of all materials in accordance with local, state, and federal regulations. Decontaminate and dispose of all potentially biohazardous material.

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EN English

Intended Use

The Face Tent is intended as an oxygen delivery alternative for patient less tolerant of oxygen masks or cannulas.

Directions for Use

1. Attach one end of corrugated tubing to face mask adapter. Connect the other end to appropriate 22 mm O.D. adapter or fixture to achieve desired setup.
2. Place mask on patient's face with patient's chin resting on bottom portion of mask. Place strap above the ears, across the back top portion of the head.

Storage

Recommended normal storage temperature 0°C/+32°F to +50°C/+122°F.

Contains Phthalates (DEHP). Risks and Precautionary Measures related to Phthalates: This instruction

AU REP AirLife Australia Holdings Pty Ltd
PO Box 97
North Ryde BC, NSW, 1670
Australia

AirLife
2710 Northridge Dr. NW, Suite A
Grand Rapids, MI 49544 USA
www.myAirLife.com
Made in Mexico

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