


REF 001269UC

AirLife™

Pediatric Oxygen Mask

Vinyl, Under the Chin Style, High Concentration, Non-rebreather Mask with Safety Vent, 7' (2.1 m) Supply Oxygen Tubing and U/Connect-It Adapter

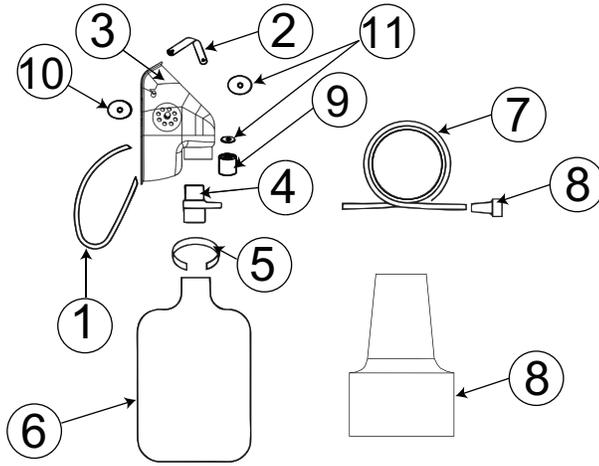

**REF** Catalogue number **Rx ONLY** U.S. Federal law restricts this device to sales by or on the order of a physician

**LOT** Lot Number **Clean, Ready to Use** Clean, Ready to Use

**QTY** Quantity **Manufacturer** Manufacturer

**LATEX** Not made with natural rubber latex **Use-by Date** Use-by Date

**PHT DEHP** Contains Phthalates **Do not re-use** Do not re-use  
Single Use

**Consult instructions for use** Consult instructions for use **Caution** Caution

**Temperature Limit** Temperature Limit

EN English

Oxygen Mask consists of:

- ① Elastic Strap
- ② Noseclip
- ③ Pediatric Oxygen Mask
- ④ Oxygen Bag Connector
- ⑤ Green Tape
- ⑥ Oxygen Bag
- ⑦ 80" Clear Vinyl Tubing 0.260"
- ⑧ U/Connect-It Adapter
- ⑨ One way Valve Adapter
- ⑩ One way Valve
- ⑪ Small One way Valve

AU REP AirLife Australia Holdings Pty Ltd  
PO Box 97  
North Ryde BC, NSW, 1670  
Australia

**AirLife**  
2710 Northridge Dr. NW, Suite A  
Grand Rapids, MI 49544 USA  
www.myAirLife.com  
Made in Mexico



Intended Use

Intended for oxygen to be delivered to the patient at a prescribed concentration and/or flow rate. Oxygen tubing is connected to an oxygen source on one end and a mask on the other end to deliver oxygen to the patient.

Directions for Use

1. Attach U/Connect-It Adapter of oxygen tubing directly to threaded oxygen outlet. (Separate tubing adapter may be used but it is not required.) Push and twist to seal. Adjust liter flow as prescribed by physician.
2. Place mask over patient's face covering both nose and mouth.
3. Slip strap over patient's head and adjust for comfort and fit.
4. Pinch nose clip to provide a seal around the nose.



Caution

- Make sure bag is inflated before attaching to patient.
- Federal (USA) law restricts this device to sale by or on the order of a physician or licensed practitioner.



Warning

- Assure adequate and uninterrupted oxygen flow to patient. If oxygen flow is inadequate or interrupted, patient may not be able to inhale, or may not be able to inhale adequate volume.
- Avoid kinking or occluding tubing.

Note

- Assure adequate and uninterrupted oxygen flow to patient. If oxygen flow is inadequate or interrupted, patient may not be able to inhale, or may not be able to inhale adequate volume.
- The oxygen connector in mask should not be connected directly to any endotracheal or tracheostomy.

Storage

Recommended normal storage temperature 0°C/+32°F to +50°C/+122°F.

Contains Phthalates (DEHP). Risks and Precautionary

Measures related to Phthalates: This instruction pertains to the phthalate symbol marked on the device or its packaging. If this device is used for the treatment of children, or treatment of pregnant or nursing women; please note that the following types of procedures may increase the risk of exposure to phthalates: Exchange transfusion in neonates, total parenteral nutrition in neonates, multiple procedures in sick neonates, haemodialysis in peripuberal males, male foetus and male infant of pregnant women, and lactating women; and massive blood infusion into trauma patients. Although these procedures have the potential for

increased risk of exposure, conclusive evidence of human health risks has not been established. As a precautionary measure, to reduce the potential for unnecessary exposures to phthalates, the product must be used in accordance with the instructions for use, and practitioners should refrain from using this product beyond the period of time the product is medically necessary or needed.

Dispose of all materials in accordance with local, state, and federal regulations. Decontaminate and dispose of all potentially biohazardous material.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer at productquality@myAirLife.com.