
REF 001363

AirLife™

Adult Venturi Style Mask

Vinyl, Under-the-Chin Style • For Single Patient Use

REF Catalogue number  U.S. Federal law restricts this device to sales by or on the order of a physician

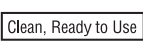LOT Lot Number  Clean, Ready to Use

QTY Quantity  Manufacturer

 Product and packaging are not made with natural rubber latex  Expiration Date

 Contains Phthalates  Temperature Limit

 Consult instructions for use  Caution

Color Code	Percent Conc.	Low Flow O ₂ Settings	Total Gas Delivered at Low Flow Settings	High Flow O ₂ Settings	Total Gas Delivered at High Flow Settings
Blue	24%	2 LPM	53 LPM	4 LPM	105 LPM
Yellow	28%	4 LPM	45 LPM	6 LPM	68 LPM
White	31%	6 LPM	47 LPM	8 LPM	63 LPM
Green	35%	8 LPM	45 LPM	10 LPM	56 LPM
Pink	40%	8 LPM	33 LPM	12 LPM	50 LPM
Orange	50%			12 LPM	32 LPM

O₂ Diluters are accurate to ± 5%

Humidity Cap Adapter

- If additional humidity is needed, remove oxygen connector from jet diluter and place nebulizer hood onto diluter jet.
- Connect a large bore corrugated tubing to 7/8 inch (22 mm) O.D. connector arm and the other end to a humidity source.

Caution

Nebulizer must be driven by compressed air—not oxygen. Use of oxygen will change the concentration of oxygen to patient. Leave humidity cup on at all times to prevent occlusion of jet particularly when patient is lying down.

Storage

Recommended normal storage temperature 0°C/+32°F to +50°C/+122°F.

Dispose of all materials in accordance with local, state, and federal regulations. Decontaminate and dispose of all potentially biohazardous material.

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English

Intended Use

Intended for oxygen to be delivered to the patient at a prescribed concentration and/or flow rate. Oxygen tubing is connected to an oxygen source on one end and a mask on the other end to deliver oxygen to the patient.

Contents: Adult Mask, 6 inch (15 cm) Flextube, Six Diluter Jets, Nebulizer Hood, 7 feet (2.1 m) Oxygen Tubing and U/Connect-It Adapter

Directions for Use

- Select appropriate diluter jet (see chart below) per physician's instructions.
- Attach desired jet to corrugated tube, attach other end of corrugated tube to mask connector.
- Attach oxygen tubing to diluter jet. Attach other end of oxygen tubing to oxygen outlet and adjust flow according to chart below.
- Place mask over patient's face covering both nose and mouth.
- Slip elastic band over patient's head and adjust for comfort and fit. For best comfort, place straps below patient's ears.
- Pinch noseclip to provide a seal around the nose.

AU REP AirLife Australia Holdings Pty Ltd
PO Box 97
North Ryde BC, NSW, 1670
Australia

 AirLife
2710 Northridge Dr. NW, Suite A
Grand Rapids, MI 49544 USA
www.myAirLife.com
Made in Mexico

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