
REF 001410

AirLife™

Corrugated Flexible EVA Tubing

For Aerosol and IPPB Use, Segmented, 6 feet (1.8 m) Length, 22 mm I.D. • For Single Patient Use

REF Catalogue number  U.S. Federal law restricts this device to sales by or on the order of a physician

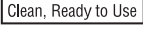LOT Lot Number  Clean, Ready to Use

QTY Quantity  Manufacturer

 Not made with natural rubber latex  Date of Manufacture

 Not made with DEHP  Caution

 Consult instructions for use

EN

English

Intended Use

Intended for oxygen to be delivered to the patient at a prescribed concentration and/or flow rate. Oxygen tubing is connected to an oxygen source on one end and to a patient interface on the other end to deliver oxygen to the patient.

Directions for Use

Tubing has segmented cuffs every 6 inches (15 cm) and may be cut to desired length. Connect tubing to any standard 22 mm O.D. fixture or adapter in accordance with directions for use of corresponding equipment.

Caution

Do not use tubing where gas temperature exceeds 154°F/68°C.

Warning

Oxygen, while not flammable, supports combustion. Do not use near excessive heat or flames, or other sources of ignition.

Dispose of all materials in accordance with local, state, and federal regulations. Decontaminate and dispose of all potentially biohazardous material.

 **AirLife**
2710 Northridge Dr. NW, Suite A
Grand Rapids, MI 49544 USA
www.myAirLife.com

AU **REP** **AirLife Australia Holdings Pty Ltd**
PO Box 97
North Ryde BC, NSW, 1670
Australia

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