Vital Signs™ Anesthesia Ventilator Circuit

REF	Catalogue number	R_X ONLY	U.S. Federal law restricts this device to sales by or on the order of a physician
LOT	Lot Number		
QTY	Quantity		Manufacturer
	Not made with natural rubber latex		Single Use
	Consult Instructions for Use		

EN

English

Instructions for Use:

To Connect:

Attach the end without the filter to the 22mm male bellows connector on the anesthesia ventilator. Attach the end with the filter to the 22mm male bag mount fitting on the anesthesia machine absorber.

To Disconnect:

Twist and pull both green cuffs off the machine to remove the Ventilator Circuit from the machine.

Test Circuit Prior to Use:

The manufacturer's instructions for the mechanical ventilator should be read, and the procedures for testing followed. If there are no testing procedures given for the circuit, test as follows:

1. Attach a circle breathing circuit to the anesthesia machine, but place the breathing bag on the patient port.
2. Close exhaust valve.
3. Turn the selector valve to the ventilator position.
4. Pressurize the circuit to 30 cm H₂O pressure for 30 seconds to ensure that there are not leaks.
5. Open exhaust valve and return the breathing bag to its proper position for use.

Please Note: Leakage may be caused by the equipment or by the patient breathing circuit.

Warning: Do not rinse, soak, wash or sterilize.

 AirLife,
2710 Northridge Dr. NW, Suite A
Grand Rapids, MI 49544 USA
www.myAirLife.com

AU **REP** AirLife Australia Holdings Pty Ltd
PO Box 97
North Ryde BC, NSW, 1670
Australia

Single Use This single-use product is not designed or validated to be reused. Reuse may cause a risk of cross-contamination, affect the measurement accuracy, system performance, or cause a malfunction as a result of the product being physically damaged due to cleaning, disinfection, re-sterilization, or reuse.

A discontinued use of this product on the same patient, with periods of use separated by a period of non-use, is acceptable. The user must ensure that the product is not damaged or contaminated between usages on the same patient.

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