REF 001207

AirLife™

Adult Aerosol Mask

Vinyl, Under-The-Chin Style High Concentration • For Single Patient Use

REF	Catalogue number	Rx ONLY	U.S. Federal law restricts this device to sales by or on the order of a physician
LOT	Lot Number	Clean, Ready to Use	Clean, Ready to Use
QTY	Quantity	Manufacturer	Manufacturer
LATEX	Not made with natural rubber latex	Expiration Date	Expiration Date
PHT DEHP	Contains Phthalates	Temperature Limit	Temperature Limit
i	Consult instructions for use		

EN

English

Intended Use

Aerosol Masks are intended to provide the patient with the means of receiving aerosol. Drainage bags are devices intended to trap and drain water that collects in tubing during respiratory therapy, thereby preventing an increase in breathing resistance.

Directions for Use

1. Attach one end of corrugated tubing to the mask and the other end to the nebulizer outlet.
2. Place mask over patient's face, covering both nose and mouth.
3. Slip elastic band over patient's head and adjust for comfort and fit. For best comfort, place straps below patient's ears.
4. Pinch noseclip to provide a seal around the nose.

Storage

Recommended normal storage temperature 0°C/+32°F to +50°C/+122°F.

Contains Phthalates (DEHP). Risks and Precautionary Measures related to Phthalates: This instruction pertains to the phthalate symbol marked on the device or its packaging. If this device is used for the treatment of children, or treatment of pregnant or nursing women; please note that the following types of procedures may increase the risk of exposure to phthalates: Exchange

transfusion in neonates, total parenteral nutrition in neonates, multiple procedures in sick neonates, haemodialysis in peripubertal males, male foetus and male infant of pregnant women, and lactating women; and massive blood infusion into trauma patients. Although these procedures have the potential for increased risk of exposure, conclusive evidence of human health risks has not been established. As a precautionary measure, to reduce the potential for unnecessary exposures to phthalates, the product must be used in accordance with the instructions for use, and practitioners should refrain from using this product beyond the period of time the product is medically necessary or needed.

Dispose of all materials in accordance with local, state, and federal regulations. Decontaminate and dispose of all potentially biohazardous material.

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