


REF 001250U

AirLife™

Pediatric Venturi Style Mask

Vinyl, Under-the-Chin Style • For Use with 50 psi Oxygen Source and Flowmeter and U/Connect-It Adapter

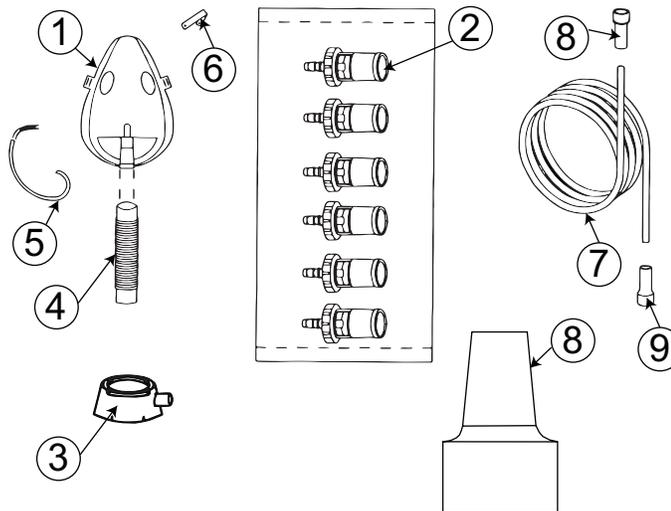

REF Catalogue number

Rx ONLY U.S. Federal law restricts this device to sales by or on the order of a physician

LOT Lot Number

Clean, Ready to Use

QTY Quantity



~~LATEX~~ Product and packaging are not made with natural rubber latex

Use-by Date

~~PHTH~~
~~DEHP~~ Contains Phthalates

Do not re-use
Single Use

Consult instructions for use

Caution

TEMPERATURE LIMIT

Intended Use

Intended for oxygen to be delivered to the patient at a prescribed concentration and/or flow rate. Oxygen tubing is connected to an oxygen source on one end and a patient interface on the other end to deliver oxygen to the patient.

Directions for Use

1. Select appropriate diluter jet (see chart below) per physician's instructions.
2. Attach desired jet to corrugated tube, attach other end of corrugated tube to mask connector.
3. Attach vinyl tip of oxygen tubing to diluter jet. Attach U/Connect-It Adapter of oxygen tubing directly to threaded oxygen outlet. (Separate tubing adapter may be used but it is not required.) Push and twist to seal. Adjust flow according to flow rate chart.
4. Place mask over patient's face covering both nose and mouth.
5. Slip strap over patient's head and adjust for comfort and fit
6. Pinch nose clip to provide a seal around the nose.

Color Code	Percent Conc.	Low Flow O ₂ Settings	Total Gas Delivered at Low Flow Settings	High Flow O ₂ Settings	Total Gas Delivered at High Flow Settings
Blue	24%	2 LPM	53 LPM	4 LPM	105 LPM
Yellow	28%	4 LPM	45 LPM	6 LPM	68 LPM
White	31%	6 LPM	47 LPM	8 LPM	63 LPM
Green	35%	8 LPM	45 LPM	10 LPM	56 LPM
Pink	40%	8 LPM	33 LPM	12 LPM	50 LPM
Orange	50%			12 LPM	32 LPM

O₂ Diluters are accurate to ± 5%

Humidification Hood

1. If additional humidity is needed, remove oxygen connector from jet diluter and place humidification hood onto diluter jet.
2. Connect a large bore corrugated tubing to 22 mm O.D. connector arm and the other end to a humidity source.

Caution

- When adding humidification using a nebulizer, the nebulizer must be driven by compressed air, not oxygen. Use of oxygen will change the concentration of oxygen to patient. Leave humidification hood in place at all times to prevent occlusion of jet.

Warning

- Avoid kinking or occluding tubing.

Note

- Assure adequate and uninterrupted oxygen flow to patient. If oxygen flow is inadequate or interrupted, patient may not be able to inhale, or may not be able to inhale adequate volume.
- The oxygen connector in mask should not be connected directly to any endotracheal or tracheostomy.

Storage

Recommended normal storage temperature 0°C/+32°F to +50°C/+122°F.

Contains Phthalates (DEHP). Risks and Precautionary Measures related to Phthalates: This instruction pertains to the phthalate symbol marked on the device or its packaging. If this device is used for the treatment of children, or treatment of pregnant or nursing women; please note that the following types of procedures may increase the risk of exposure to phthalates: Exchange transfusion in neonates, total parenteral nutrition in neonates, multiple procedures in sick neonates, haemodialysis in peripubertal males, male foetus and male infant of pregnant women, and lactating women; and massive blood infusion into trauma patients. Although these procedures have the potential for increased risk of exposure, conclusive evidence of human health risks has not been established. As a precautionary measure, to reduce the potential for unnecessary exposures to phthalates, the product must be used in accordance with the instructions for use, and practitioners should refrain from using this product beyond the period of time the product is medically necessary or needed.

Dispose of all materials in accordance with local, state, and federal regulations. Decontaminate and dispose of all potentially biohazardous material.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer at productquality@myAirLife.com.

EN English

Venturi Style Mask consists of:

- 1 Pediatric Mask
- 2 6 Color Coded Diluter Jets
- 3 Humidification Hood
- 4 6 inch (15 cm) Flextube
- 5 Elastic Strap
- 6 Noseclip
- 7 7 feet (2.1 m) Crush Resistant Oxygen Tubing
- 8 U/Connect-It Adapter (Threaded Adapter)
- 9 Vinyl Tip Adapter

AirLife
2710 Northridge Dr. NW, Suite A
Grand Rapids, MI 49544 USA
www.myAirLife.com

