REF 001504

AirLife™

Ventilator Monitoring Adapter Circuit

Contains U/Adapt-It with 15 mm ID Base, 6 inch (15 cm) Flex Tube, Two One-Way Valves, Oxygen Tube Connector, 22 mm O.D. Tube Adapter, and 22 mm I.D./15 mm I.D. Adapter • For Single Patient Use

REF Catalogue number

U.S. Federal law restricts this device to sales by or on the order of a physician

LOT Lot Number

Clean, Ready to Use

Clean, Ready to Use

QTY Quantity

Manufacturer

Not made with natural rubber latex

Date of Manufacture

Not made with DEHP

Caution

Consult instructions for use

EN

English

Intended Use

The Monitoring Circuit's intended use is to provide the components needed to attach monitoring devices to patients' endotracheal or tracheostomy tube.

Directions for Use


1. This one-way valving system is designed for pulmonary function measurements such as tidal volume, minute volume, vital capacity, and negative inspiratory force. The one-way valves permit only unidirectional flow of gases to prevent cross-contamination from the device used for measurement.
2. Expired volume measurements are performed by connecting any portable respirometer to the expiratory limb of this circuit (utilizing adapters provided).
3. Connect 15 mm ID base of trach tee to 15 mm patient trach or endotracheal tube adapter.
4. To perform negative inspiratory force measurements, attach any negative pressure measurement device to the inspiratory side of the trach tee.
 - A. Connect 15 mm ID base of trach tee to 15 mm patient trach or endotracheal tube adapter.

- B. Connect a short piece of standard drive line tubing to measurement device.
- C. Connect other end of drive line tubing to oxygen connector provided.
- D. Assemble oxygen connector with tubing to the inspiratory side of trach tee after removing one-way valve on inspiratory side of U/Adapt-It.
- E. Oxygen monitor stem is provided for additional oxygen or as a monitoring system.
- F. Thermometer port is provided for use with disposable thermometers, thermistor probe, or suctioning port.

Note: Insert oxygen connector until snug only.

Cautions

- Do not leave patient unattended while using this device.
- The two flutter valves are assembled facing in the same direction. Always check valve positions to be sure that the valves are properly placed for proper inspiration and expiration before placing on patient.

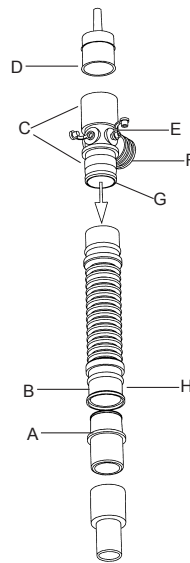- A. 22 mm I.D./15mm I.D. Adapter
- B. Expiratory Side
- C. One-Way Valves
- D. Oxygen Connector
- E. Inspiratory Side
- F. Trach Tee Base
- G. Direction of Flow
- H. 22 mm O.D. Tube Adapter

Dispose of all materials in accordance with local, state, and federal regulations. Decontaminate and dispose of all potentially biohazardous material.

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