REF 001505

AirLife™

Tee Adapter • With One Arm 22 mm I.D.,
One Arm 22 mm O.D., 18 mm I.D. Base
with Mouthpiece

REF Catalogue number **R_X ONLY** U.S. Federal law restricts this device to sales by or on the order of a physician

LOT Lot Number **Clean, Ready to Use** Clean, Ready to Use

QTY Quantity **Manufacturer** Manufacturer

LATEX Not made with natural rubber latex **Date of Manufacture** Date of Manufacture

PHT/DEHP Not made with DEHP **Single Use** For single use only

i Consult instructions for use

EN

English

Intended Use

The Tee adapter is intended for use with small volume nebulizers to deliver aerosolized medication for inhalation.

Directions for Use

1. Insert nebulizer into the base of the tee adapter. Attach the 22 mm I.D. arm onto the 22 mm O.D. outlet of spirometer.
2. Insert mouthpiece in the other arm of tee adapter.

Note: This item was specifically designed for use with incentive spirometers having 22 mm O.D. outlets.

Dispose of all materials in accordance with local, state, and federal regulations. Decontaminate and dispose of all potentially biohazardous material.

AirLife
2710 Northridge Dr. NW, Suite A
Grand Rapids, MI 49544 USA
www.myAirLife.com

AU REP AirLife Australia Holdings Pty Ltd
PO Box 97
North Ryde BC, NSW, 1670
Australia

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