REF 001840

AirLife™

Universal Oxygen Nut and Nipple Connection

For Single Patient Use

REF Catalogue number **R_X ONLY** U.S. Federal law restricts this device to sales by or on the order of a physician

LOT Lot Number **Clean, Ready to Use** Clean, Ready to Use

QTY Quantity **Manufacturer** Manufacturer

LATEX Not made with natural rubber latex **Date of manufacture** Date of manufacture

PHT/DEHP Not made with DEHP **Consult instructions for use** Consult instructions for use

EN

English

Intended Use

The universal Nut and Nipple is intended for use in the connection of oxygen delivery tubing to an oxygen flowmeter or regulator with a DISS male outlet.

Directions for Use

1. Attach threaded end of connector securely to flow meter.
2. Connect tubing connector to outlet stem.
3. Administer oxygen per physician's orders.

Warning

Oxygen, while not flammable, supports combustion. Do not use near excessive heat, flames, or other sources of ignition.

Dispose of all materials in accordance with local, state, and federal regulations. Decontaminate and dispose of all potentially biohazardous material.

AirLife
2710 Northridge Dr. NW, Suite A
Grand Rapids, MI 49544 USA
www.myAirLife.com

AU REP AirLife Australia Holdings Pty Ltd
PO Box 97
North Ryde BC, NSW, 1670
Australia

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