REF 002003

AirLife®

Bubble Humidifier

With 3 psi Pressure Relief Valve •
For Single Patient Use

REF	Catalogue Number	R_X ONLY	U.S. Federal law restricts this device to sales by or on the order of a physician
LOT	Lot Number	Clean, Ready to Use	Clean, Ready to Use
QTY	Quantity		Manufacturer
	Not made with natural rubber latex		Date of Manufacture
	Not made with DEHP		For Single Use Only
	Consult Instructions for Use		Caution

EN

English

Intended Use

This humidifier's intended use is for humidification of supplemental oxygen being delivered to a spontaneously breathing patient via oxygen delivery device.

Directions for Use

1. Fill the humidifier jar to the maximum fill line with sterile water.
2. Tighten cap securely to bottle in order to create an effective seal.
3. Attach the wing nut to the oxygen source.
4. Test for leaks and proper function of pressure relief valve whistle. Apply gas flow of 3 L/min. Occlude O₂ port and listen for alarm whistle. If alarm does not activate, check all connections for possible leaks.
5. Connect supply tubing (not included) to the small barbed outlet connector.
6. Adjust oxygen flow as directed by physician. Verify that bubbles are present.

Note:

- Do not overfill. Do not allow the water to fall below the minimum fill line during use.
- Alarm will actuate between 2.7 and 3.3 psi (18.6 and 22.8 kPa). Flow rates of less than 2 L/min. may affect the audible alarm.

Caution

This unit is equipped with a pressure relief valve whistle

AirLife
2710 Northridge Dr. NW, Suite A
Grand Rapids, MI 49544 USA
www.myAirLife.com

AirLife Australia Holdings Pty Ltd
PO Box 97
North Ryde BC, NSW, 1670
Australia

that will activate when excess pressure develops inside the bottle. If the whistle sounds, turn off the oxygen flowmeter. Examine the mask, cannula or oxygen tubing for restrictions. Failure to correct an obstruction could interrupt the flow of oxygen and may cause harm to the user.

Dispose of all materials in accordance with local, state, and federal regulations. Decontaminate and dispose of all potentially biohazardous material.

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