REF 0Y1777

AirLife™

R_X ONLY Clean, Ready to Use**EN Ventilator Circuit 2.0 m (6.5')**

With (3 mm) 1/8" and (6 mm) 1/4" Pressure Lines and Valve Tee

For use with ventilators that have internal exhalation valves.

Contents

Ventilator Circuit with Two (2.0 m) 6.5' Corrugated Tubes
 Ported Patient "Y" with Elbow
 Tube Hanger
 Two Hose Clips
 Humidifier Adapter with Temperature Port
 (3 mm) 1/8" and (6 mm) 1/4" Pressure Lines
 Valve Tee

Rated Flow: 70 l/min, ≤ 0.2 kPa (2.0 cm H₂O)**Directions for Use**


1. **Inspiratory Line:** Connect the inspiratory line to the ventilator outlet port or humidifier chamber outlet port (if used).
2. **Expiratory Line:** Connect the expiratory line to the exhalation port of the ventilator.
3. **Pressure Line:** The pressure-sensing line connects the pressure port of the patient "Y" to the pressure port on the ventilator. If the line is not required, use the appropriate plug to seal the pressure port on the patient "Y".
4. **Valve Tee:** Maintain cap in closed position when not in use. Activate valve by placing nebulizer into the base of the tee adapter. To ensure a tight fit, press and twist the nebulizer firmly into place. Once the procedure is completed, twist and pull to remove the nebulizer. Reclose cap.
5. If temperature control and/or monitoring is required, install temperature probe (not included) into the temperature ports, removing any plugs prior to inserting the probe.
6. Ensure that all connections are tight. Some connections are deliberately assembled only finger tight and must be firmly secured.
7. Test circuit prior to use by occluding the patient connection port, and pressure test the circuit to ensure that there are no leaks. Also, check for occlusions by allowing the gas to flow, and ensuring that gas is emitted from the patient connection port. Block the patient connection port and ensure that gas is flowing through the expiratory limb.

Caution: Do not use this circuit where gas temperature at the outlet of the humidifier exceeds 154°F/68°C. Be aware that the compliance and flow resistance characteristics of the circuit may affect the operating parameters and capabilities of the ventilator.

Caution: For single use only. Re-use may degrade the performance of the product or contribute to cross contamination.

Warnings: Do not soak, rinse, wash, sterilize, or reuse this product.

Contains Phthalates (DEHP). The benefit of treatment with medical respiratory support devices outweighs the remote possibility of exposure to phthalates.

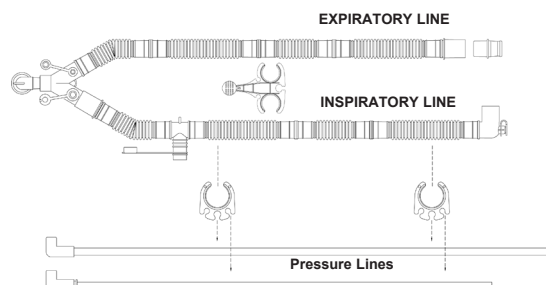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AirLife Australia Holdings PTY LTD