AirLife™ HYGROSCOPIC CONDENSER HUMIDIFIER

REF	Catalogue Number		U.S. Federal law restrict this device to sales by or on the order of a physician
LOT	Lot Number		Manufacturer
QTY	Quantity		For Single Use Only
	Consult Instructions for Use		Date of Manufacture
	Not made with natural rubber latex		Warning

EN

English

Indications for Use: A heat and moisture condenser (artificial nose) is a device intended to be positioned over a tracheotomy (a surgically created opening in the throat) or tracheal tube (a tube inserted into the trachea) to warm and humidify gases breathed in by a patient.

Directions for Use:

Connect HCH device (with or without AirLife flexible elbow) securely to the endotracheal tube, tracheostomy tube, face mask and/or breathing circuit (as applicable). Pressure check system to ensure leak free connections. Replace every 24 hours, or more often if needed to prevent accumulation of secretions. When using the HCH, the ventilation parameter and arterial blood gases should be monitored closely. Compensation for internal dead space of the HCH device (and flexible elbow if used) may be necessary.

Note: Due to the HCH's high efficiency, you may see condensate in the polybag. This is a normal occurrence due to the high efficiency of the HCH product. If tracheostomy HCH – Attach oxygen tubing, if used, to flow meter as prescribed by physician.

Do Not Use an HCH Device in the following Situations:

1. In patients who cannot tolerate the additional mechanical dead space of this device.
2. In patients who cannot tolerate the additional airway resistance of this device.
3. In patients with voluminous secretions.

Warning: Do not rinse, soak, wash or sterilize. Do not add moisture or use in conjunction with heated water humidifiers or nebulizers. Do not allow saline to enter HCH device when suctioning patients as resistance may temporarily increase. Due to increased resistance of the device, the patient must be closely monitored with proper alarms turned on. If any complications are observed, such as mucous plugging, proper airway care must be instituted immediately.

Dispose of all materials in accordance with local, state, and federal regulations. Decontaminate and dispose of all potentially biohazardous material.

This single-use product is not designed or validated to be reused. Reuse may cause a risk of cross-contamination, affect the measurement accuracy, system performance, or cause a malfunction as a result of the product being physically damaged due to cleaning, disinfection, re-sterilization, or reuse.

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