EN Volume Ventilator Circuit 1.6 m (5')

For Portable Ventilator Machines.

Compatible with Life Products LP-4, LP-5*, LP-6

Lifecare PLV-100**, PLV-102

Aequitron LP-6, LP-6 Plus and LP-10

Rated Flow: 13 l/min, ≤ 0.2 kPa (2.0 cm H₂O)**R_X**_{ONLY}

Clean, Ready to Use

Directions for Use


1. Attach elbow (A) with corrugated tubing to ventilator patient air outlet. Attach other end to inlet side of humidifier, if used.
2. Attach end of (1.6 m) 5' corrugated tubing (B) to outlet side of humidifier or patient air outlet as appropriate.
3. Attach one end of exhalation valve supply line (C) to exhalation valve stem. Attach other end of supply line to "exhalation valve" outlet connector on ventilator.
4. Attach one end of pressure sensing line (D) to "pressure sensing" stem on manifold body. Attach other end of pressure sensing line to "pressure sensing" outlet connector on ventilator. **For Life Products LP-5, LP-6 and Aequitron LP-6, LP-6 Plus and LP-10 ventilators only:** cut the entire white connector off of the blue pressure line before attaching to the ventilator.
5. On ventilators without external pressure sensing, leave the cap on the port on the manifold body.
6. Remove blue end cap (E) from end of patient trach elbow with (15 cm) 6" corrugated tube. Trach elbow adapter will fit a 22 mm I.D. or 15 mm O.D. adapter. Attach 15 mm I.D. arm to trach or endotracheal adapter.
7. Make all ventilator settings in accordance with physician's orders.

Caution: Before use, firmly attach exhalation valve supply line to exhalation stem and exhalation valve outlet. Insure that all connections are properly secured.

Warning: Ventilator monitoring and warning systems should be used as specified by ventilator manufacturer.

Contains Phthalates (DEHP). The benefit of treatment with medical respiratory support devices outweighs the remote possibility of exposure to phthalates.

Caution: For single use only. Re-use may degrade the performance of the product or contribute to cross contamination.

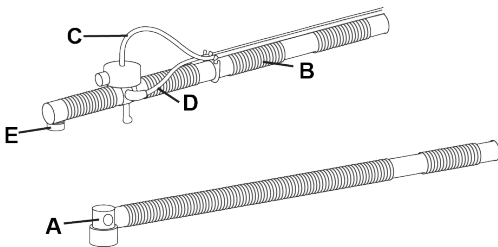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*Life Products LP-4 and LP-5 are trademarks of Lifecare International, Inc.
**Lifecare PLV-100 is a trademark of Abbott Laboratories.
AirLife Australia Holdings PTY LTD