Vital Signs™ Laryngoscope Blade

For exclusive use with Vital Signs
Laryngoscope Handles
Instructions for Use

REF Catalogue number

U.S. Federal law
restricts this
device to sales
by or on the order of
a
physician

LOT Lot Number

Manufacturer

QTY Quantity

Single Use

LATEX Not made with
natural rubber latex

Date of
Manufacture

i Consult
Instructions
for Use

Use-by Date

EN

English

Intended Use: Vital Signs blades are to be used in endotracheal intubations during anesthesia, intensive care, and emergency medical airway management.

The Vital Signs Sheath is offered as an accessory to the Laryngoscope Handles and Lighted Intubation Stylet Handles. It covers the handles to reduce the possibility of fluids from coming in contact with them.

Note: If used with a Vital Signs Laryngoscope Handle Sheath, begin with the Vital Signs Laryngoscope Handle Sheath instructions below.

Instructions for Use:

1. With the blade still in its polybag packaging, grasp the blade flange. Then open polybag at heel end of blade and expose sufficient blade to attach handle.
2. Attach heel of blade to handle hinge, pressing firmly until audible confirmation.
3. Lift and engage blade. Verify illumination.
4. Return blade to stand-by position. Remove polybag prior to intubation.
5. Dispose of blade according to standard hospital practice.

Vital Signs Laryngoscope Handle Sheath

For use with laryngoscope handle.

1. Insert the bulb end of the Laryngoscope Handle between the white plastic strip and plastic layer attached to paper backing sheet.
2. While holding the white plastic strip and paper backing, peel down the blue plastic strip separating it from the handle sheath.
3. While holding Laryngoscope Handle separate white paper backing sheet from sheath.
4. Attaching a Laryngoscope Blade to a Laryngoscope Handle will allow the Vital Signs Laryngoscope Handle Sheath to be punctured.

Note: After intubation, slip the Vital Signs Laryngoscope handle cover over the laryngoscope blade and dispose.

- This product should not be used/ discontinued if physical or function deterioration occurs.
- For expiration date, reference product labeling.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer at productquality@myAirLife.com and the competent authority of the Member State in which the user and/or patient is established.

Dispose in container that is clearly labeled, leak-proof, and color-coded per local standards and state environmental regulations.

This single-use product is not designed or validated to be reused. Reuse may cause a risk of cross-contamination, affect the measurement accuracy, system performance, or cause a malfunction as a result of the product being physically damaged due to cleaning, disinfection, re-sterilization, or reuse.

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