REF 001794

AirLife™

EN Volume Ventilator Circuit

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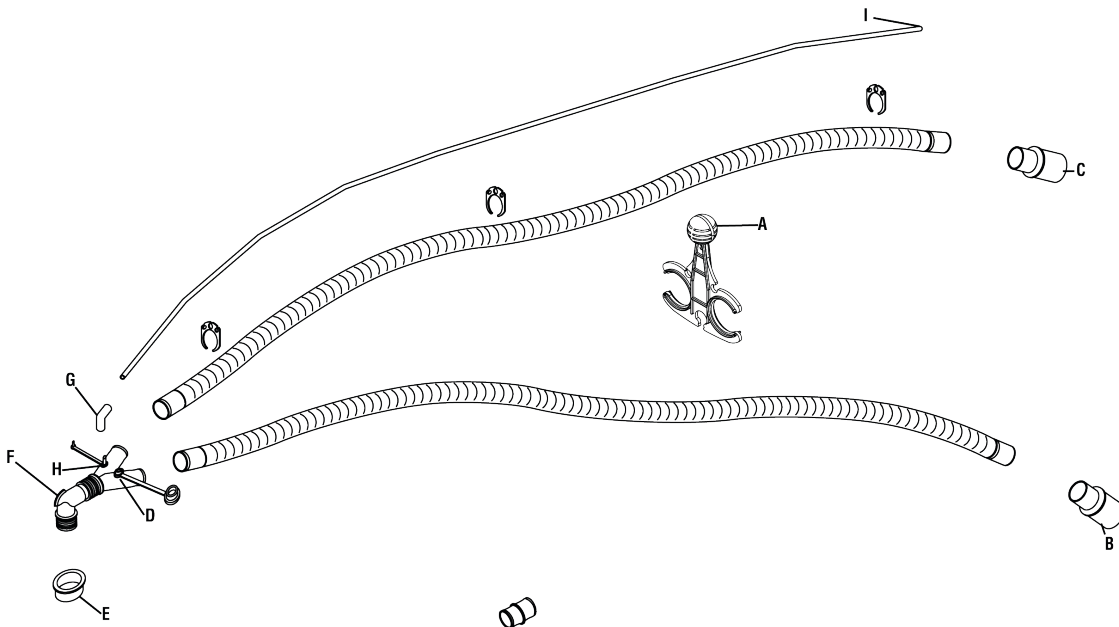
AirLife®

EN Volume Ventilator Circuit

For Use with Bear 5*, 1000, and Any Volume Ventilators Requiring 1/4 inch Pressure-Sensing Line

Contents: Two 6 foot (1.83 m) Corrugated Tubes, Ported Patient "Y" with Port Plugs, Elbow Connector, Pressure-Sensing Line, and Tube Hanger.

Directions for Use

1. Remove circuit from package.
2. Attach circuit hanger to support arm on ventilator. (A)
3. Attach inspiratory limb tubing with humidifier adapter (B) to humidifier outlet or to inspiratory port of ventilator if humidifier is not used.
4. Attach expiratory limb tubing with spirometer attachment (C) to spirometry connector on ventilator.
5. The Temp_o-I™ thermometer or an electronic thermistor may be inserted in port on left arm of "Y" (D) for proximal temperature monitoring.
6. Attach elbow on end of pressure monitoring line (G) to stem (H) on right arm of "Y." Attach other end of monitoring line to appropriate ventilator inlet. Check connections for secure fit to stem and ventilator.
7. Remove blue end cap (E). Attach trach elbow (F) to patient trach tube.
8. Make all machine settings in accordance with physician's orders.

Warning: Ventilator monitoring and warning systems should be used as specified by ventilator manufacturer.

Cautions: Before and during use, ensure that all connections are properly secured. If either port opening on the arms of the "Y" is not used, check to make sure the attached plugs are inserted tightly into the holes to prevent leakage.

Caution: For single use only. Re-use may degrade the performance of the product or contribute to cross contamination.

Hose Compliance Factor

	At Room Temp 72°F (22°C)	Heated 98.6°F (37°C)
Hose Compliance per cm	.00629 ml/cm H ₂ O	.00669 ml/cm H ₂ O

Note: To obtain the compliance of the entire circuit, multiply the total length of tubing used by the per cm compliance factors shown above.

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AirLife Australia Holdings PTY LTD