## Vital Signs™ Oral Airway Instructions for Use

|                                                                                  |                                    |                                                                                   |                                                                                   |
|----------------------------------------------------------------------------------|------------------------------------|-----------------------------------------------------------------------------------|-----------------------------------------------------------------------------------|
|  | Catalogue number                   |  | U.S. Federal law restricts this device to sales by or on the order of a physician |
|  | Lot Number                         |  | Manufacturer                                                                      |
|  | Size                               |  | Single Use                                                                        |
|  | Quantity                           |  | Date of Manufacture                                                               |
|  | Not made with natural rubber latex |  | Exp. Date                                                                         |
|  | Consult Instructions for Use       |                                                                                   |                                                                                   |

### EN English

**Intended Use:** An oral airway is a device inserted into a patient's pharynx through the mouth to provide a patent airway.

#### Berman and Guedel Airway Instructions For Use:

1. Tilt patient's head back and pull chin up to open air passage unless contraindicated by neck fracture symptoms.
2. Clear the patient's mouth and air passage of foreign matter.
3. Select the appropriate airway size.

**Warning:** Check for clear air flow passage prior to use.

#### Note:

1. Select the appropriate airway size and check for clear air flow passage prior to use to avoid incorrect therapeutic output or function loss.
2. This product should not be used or discontinue use if physical or functional deterioration, brittleness or discoloration occurs.
3. Long term use of this product may cause irritation, soreness and laceration.
4. For expiration date, reference product labeling.

This single-use product is not designed or validated to be reused. Reuse may cause a risk of cross-

contamination, affect the measurement accuracy, system performance, or cause a malfunction as a result of the product being physically damaged due to cleaning, disinfection, re-sterilization, or reuse.

Dispose in container that is clearly labeled, leak-proof, and color-coded per local standards and state environmental regulations.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer at [productquality@myAirLife.com](mailto:productquality@myAirLife.com) and the competent authority of the Member State in which the user and/or patient is established.

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