Vital Signs™ ECG Papers for GE MAC® Series

Instructions for Use

[REF] 22616702	Vital Signs™ ECG Paper, MAC® Series, 8 years Imaged Archival Life, 90mm, Z-Fold	
2030887-001	Vital Signs™ ECG Paper, MAC® Series, 8 years Imaged Archival Life, 80mm, Red Grid, Z-Fold	
2036970-001	Vital Signs™ ECG Paper, MAC® Series, 8 years Imaged Archival Life, 110mm, Red Grid, Z-Fold	
22616605	Vital Signs™ ECG Paper, MAC® Series, 8 years Imaged Archival Life, A4, Red Grid, Z-Fold	
[REF]	Catalogue number	[i] Consult instructions for use
[LOT]	Lot number	[E] Expiry date
[UDI]	Unique device identifier	[QTY] Quantity
[M]	Manufacturer	[MD] Medical device
[M]	Date of manufacture	[CE] The marking declares compliance with the applicable EU legislation.
[M]	Temperature limit	[UK] The UK Conformity Assessed marking declares compliance with the applicable UK legislation.
[M]	Humidity limitation	[SE] Indicates the authorized representative in Switzerland
[M]	Keep dry	[M] Keep away from sunlight

Intended use

ECG recorder paper is used to retain printed ECG waveforms and the associated ECG information. The paper provides a preprinted grid to facilitate measurements. ECG recorder paper shall be used by or under the direction of a licensed health care practitioner. The ECG recorder paper is intended to be used primarily in the hospital but can be used in clinics, physician offices, outreach centers or wherever ECGs are acquired.

Instructions for use

Vital Signs™ ECG Paper for GE MAC® Series:

- Provides a preprinted grid to facilitate measurements.
- Is designed specifically to match the characteristics of GE Healthcare equipment.
- This is a special fine paper that is impregnated with a chemical that changes color when exposed to heat.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer at productquality@myairlife.com and the competent authority of the Member State in which the user and/or patient is established.

Notes

Carefully visualize preprinted grid ECG paper before use. Device captures and presents data reflecting a patient's physiological condition when reviewed by a trained physician or clinician can be useful in determining a diagnosis; however, the data should not be used as a sole means for determining a patient's diagnosis.

Storage Recommendations for Imaged Paper

- Store in cool, dark, and dry locations. Temperature must be below 27°C (80°F). Relative humidity must be < 65%.
- Avoid exposure to bright light or UV sources such as sunlight, fluorescent, and similar lighting.
- Store separately in a manila folder to ensure maximum trace and patient data image life.
- Do not store with carbon and carbonless forms, plastic document protectors, or pressure sensitive tapes or labels.

Storage

Storage temperature -30°C/-22°F to +30°C/+86°F.

Storage humidity 0% - 65%.

Disposal

Dispose per local and state environmental regulations.

Technical description

- Grid is perpendicular to paper within +/- 0.5%.
- Grid ruling has a tolerance of +/- 2% per 1mm.
- Durability of a clear printout on a Vital Signs ECG Paper is readable according with the product description from the date of imaging with proper storage.

Ordering information

For further information, please contact your local sales representative or visit www.myAirLife.com.

[M]	[CE] 1297
AirLife Finland Oy Kuortaneenkatu 2 FI-00510 Helsinki, Finland www.myAirLife.com	
Made in Switzerland	
UK Responsible Person: MT Promed Consulting Ltd, First Floor, Park Central, 40-41 Park End Street Oxford OX1 1JD United Kingdom	[UK] 0086
Australian Sponsor: AirLife Australia Holdings Pty Ltd, PO BOX 97 NORTH RYDE BC NSW 1670 Australia	
MediEnvoy Switzerland, Gotthardstrasse 28 8302 Zug Switzerland	
1000005670 Rev.B 2025/02/2	

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